




Olympia Care XLow

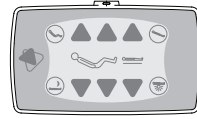

Manipulation		Hand switch	Hand switch HD	Satellite control FPP
BACKREST				
KNEEREST				
RELAX POSITION				
HIGH/LOW				
SLEEPING POSITION				
REVERSE-TREND.				
Emergency operations				
TREND.				
CPR				
Lock/Unlock				
Underbedlight				


The available bed functionality is in line with the chosen and ordered configuration.



Olympia Care XLow

Bed extension

(not for Soft-version)

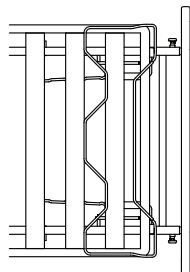

1. lift the footrest
2. pull out both pull knobs
3. pull out the bed extension
4. lock the buttons
5. pull out the foot extension (option)

2 by 2 braked castors

2 castors unbraked

2 castors braked

