



Nr.	Artikel Nr.	Benämning	Default/QTY.
1	CS154000R	Sängsarg Ryggdel 90	1
2	CS154000F	Sängsarg Fotdel 90	1
3	CS154500	Aktiv Ryggdel 90	1
4	CS153620	Vadlyft 90 kpl	1
5	CS153610	Lårlyft 90 kpl.	1
6	CS154710	Lårram 90 kpl	1
7	CS034100	Vadbåge 90 kpl. Lackad	1
8	CS154100	Gavel 90 kpl	2
9	CS151220	Länk Delbar Aktiv Rygg kpl	2
10	CS153581	Ryggglänk lång	2
11	CS154591	Ryggglänk Kort	2
12	CS154099	Madrassbygel Rygg	2
13	CS008012	Madrassbygel Vaddel	2
14	CS007101	Fothandtag Qvintett S lackad	1
15	CS007000	Länk Lår-Vadlyft	1
16		Märketikett Sängbotten	1
17		Etikett Serienummer	1
18	AA201090	Filtrondell 20x2,5 Nr 16-20 vit	4
19	AA202266	Glidbussning med fläns COB010F 25115	4
20	AA202224	Glidbussning utan fläns COB1012 08	8
21	CS009002	Exenterlås 2.30.02 ST/FZBM Lockingmutter	3
22	AA202129	Stopp-skruv M8x16 FZB S6SS 10,9	2
23	AA202017	Skruv MRT-GF M4x8 fzb	10
24	CS008003	Sprint plastgångjärn	2
25	AA202024	Spårryttare 6mm EUR fz.	2
26	AA208037	S6SS M8x12 fzb Stoppskruv	4
27	CS068905	CBH bult D12 L=50	3
28	AA202341	Låssprint LCS rund 2,5 RF SS2331	1
29	AA202021	Hårnålssprint fjädrande 2,5x42 BR	4
30	AA202186	Låsmutter M8 DIN985 stål fzb.	8
31	AA202130	Planbricka 8,4x25x2,5 Nylon	8
32	AA202225	Skruvhuvud F6N M6	2
33	AA202298	Skruv K6S M6x16 Fzb	2
34	AA202233	Skruv MFTS M6x20 8.8 FZB	4
35	AA202234	Skruv MFTS M&x30 8.8 FZB	4
36	CS008002	CBH bult D12 L=60	2
37	AA208083	Ratt M8x25	2
38	AA208084	Distans 8,2x10x12	2
39	AA208079	Mutter ML6M M8	2
40	AA208082	BRB M8 fzb 8,4x16x1,5	8
41	AA208072	Ändavslut 25x20 Svart	2
42	CS009001	Gångjärn/Hinge	4
43	CS151214	Glidkloss	4
44	AA201011	Bricka Nylon 12,8x18x3	8
45	AA201010	Distans 12,8x18x5 Nylon	1
46	CS149710	Latt 718	15
47	CS149800	Latthållare	30

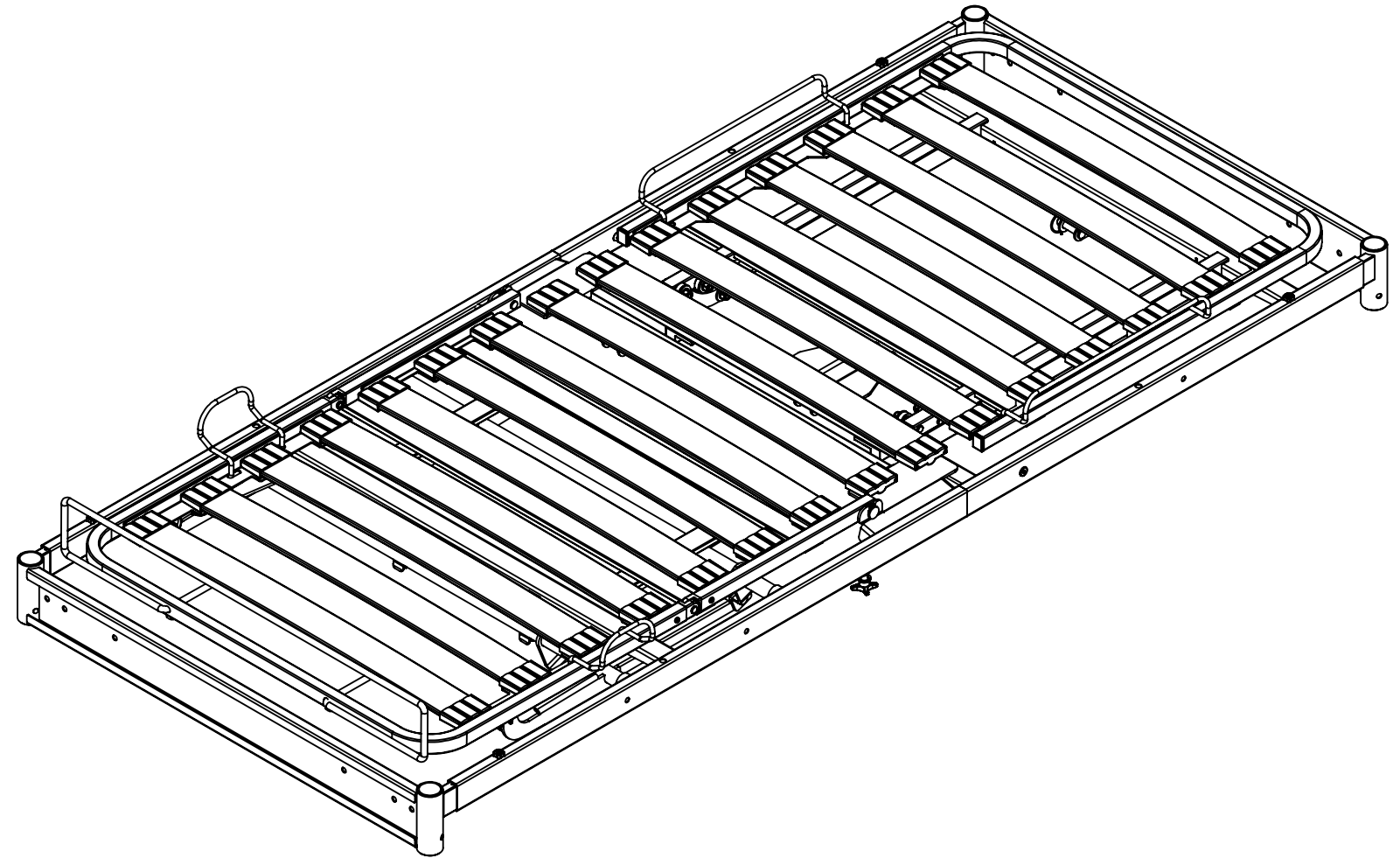

This document is the property of Comfort System Scandinavia Aktiebolag and may not be copied without our written permission, and the contents thereof must not be imparted to a third party nor be used for any unauthorized purpose. Contravention will be prosecuted.

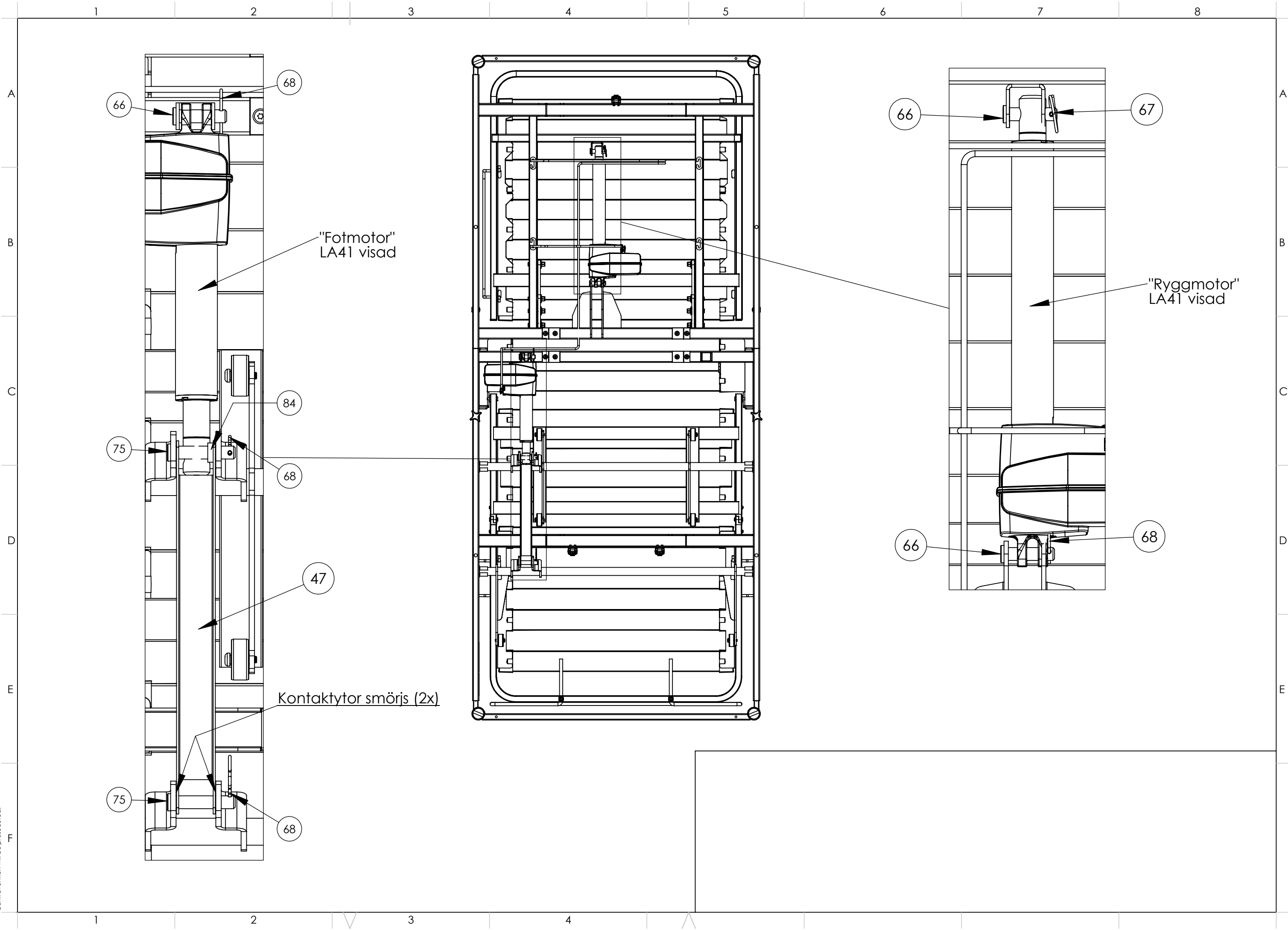

66 68

"Fotmotor"
LA41 visad

84

75

68

47

Kontaktytor smörjs (2x)

75

68

66

67

"Ryggmotor"
LA41 visad

66

68

A

B

C

D

E

F


A

B

C

D

E

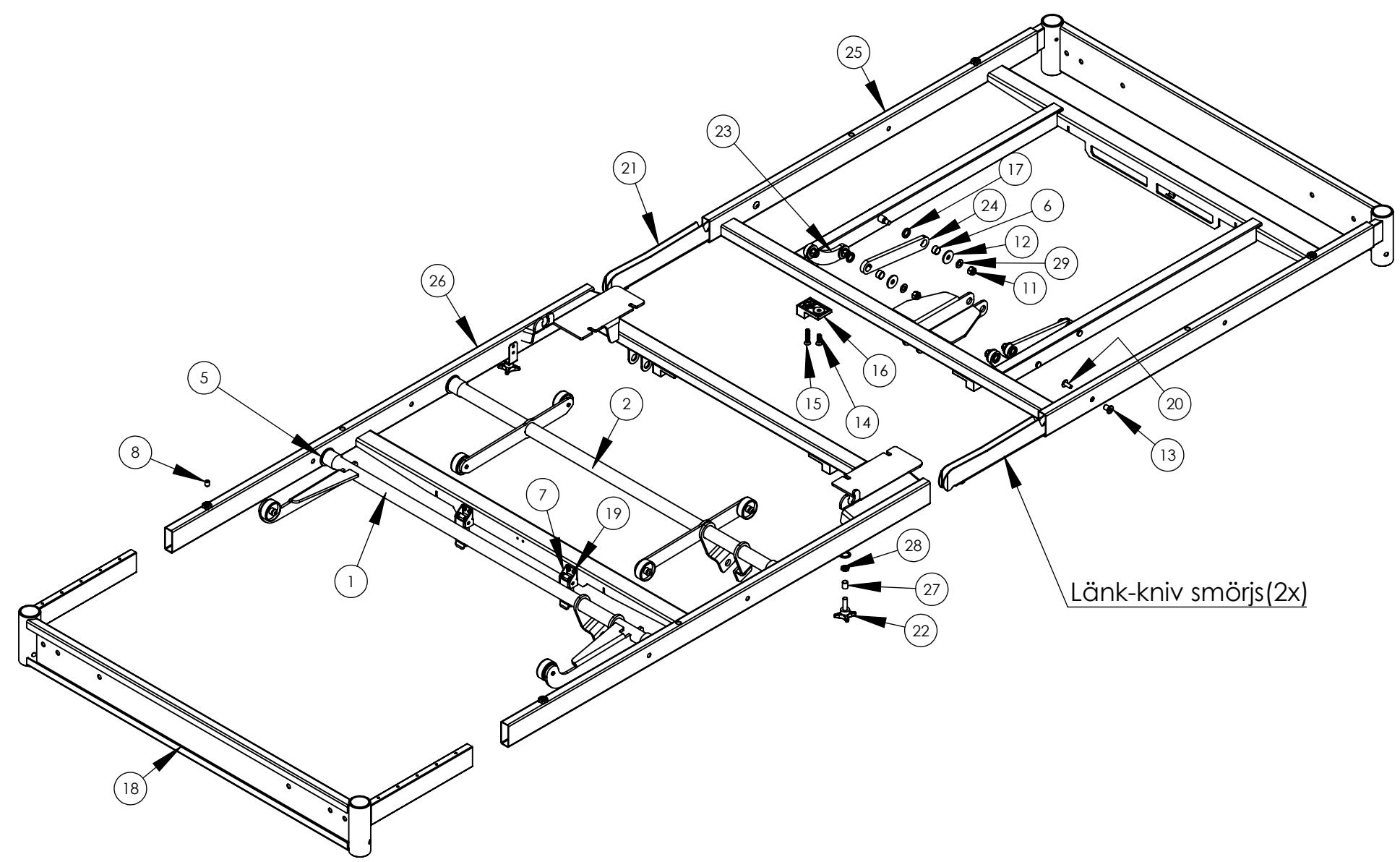

Nr.	Artikel Nr.	Benämning	Exp/QTY.
1	CS153620	Vadlyft 90 kpl	1
2	CS153610	Lårlyft 90 kpl.	1
3		Märketikett Sängbotten	1
4		Etikett Serienummer	1
5	AA202266	Glidbusning med fläns COB010F 25115	4
6	AA202224	Glidbusning utan fläns COB1012 08	8
7	CS009002	Exenterlås 2.30.02 ST/FZBM Lockingmutter	3
8	AA208037	S6SS M8x12 fzb Stoppskruv	4
9	CS068905	CBH bult D12 L=50	1
10	AA202021	Hårnålssprint fjädrande 2,5x42 BR	1
11	AA202186	Låsmutter M8 DIN985 stål fzb.	8
12	AA202130	Planbricka 8,4x25x2,5 Nylon	8
13	AA202225	Skruvhuvud F6N M6	2
14	AA202233	Skruv MFTS M6x20 8.8 FZB	4
15	AA202234	Skruv MFTS M&x30 8.8 FZB	4
16	CS151214	Glidkloss	4
17	AA201011	Bricka Nylon 12,8x18x3	8
18	CS154100	Gavel 90 kpl	2
19	AA202017	Skruv MRT-GF M4x8 fzb	6
20	AA202298	Skruv K6S M6x16 Fzb	2
21	CS151220	Länk Delbar Aktiv Rygg kpl	2
22	AA208083	Ratt M8x25	2
23	CS154591	Ryggglänk Kort	2
24	CS153581	Ryggglänk lång	2
25	CS154000R	Sängsarg Ryggdel 90	1
26	CS154000F	Sängsarg Fotdel 90	1
27	AA208084	Distans 8,2x10x12	2
28	AA208079	Mutter ML6M M8	2
29	AA208082	BRB M8 fzb 8,4x16x1,5	8

This document is the property of Comfort System Scandinavia Aktiebolag and may not be copied without our written permission, and the contents thereof must not be imparted to a third party nor be used for any unauthorized purpose. Contravention will be prosecuted.

1 2 3 4 5 6 7 8

A

B

C

D

E

F



A

B

C

D

E

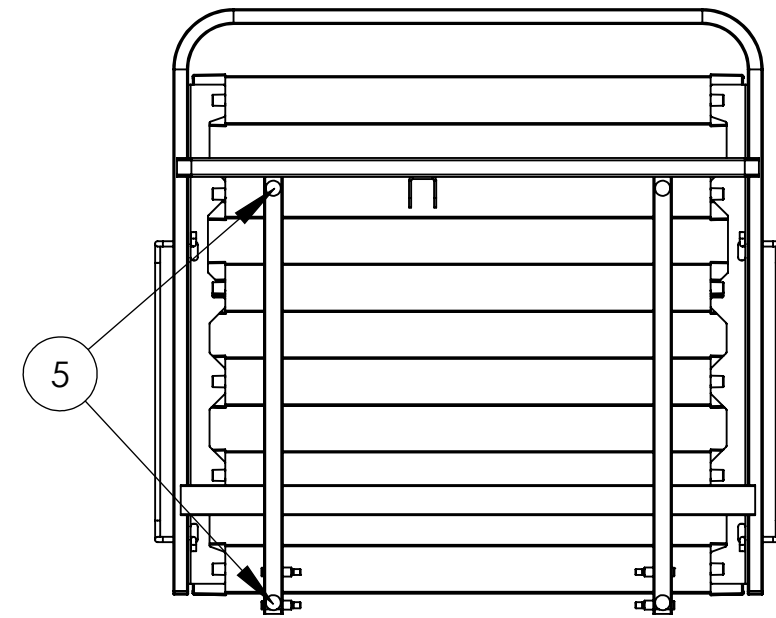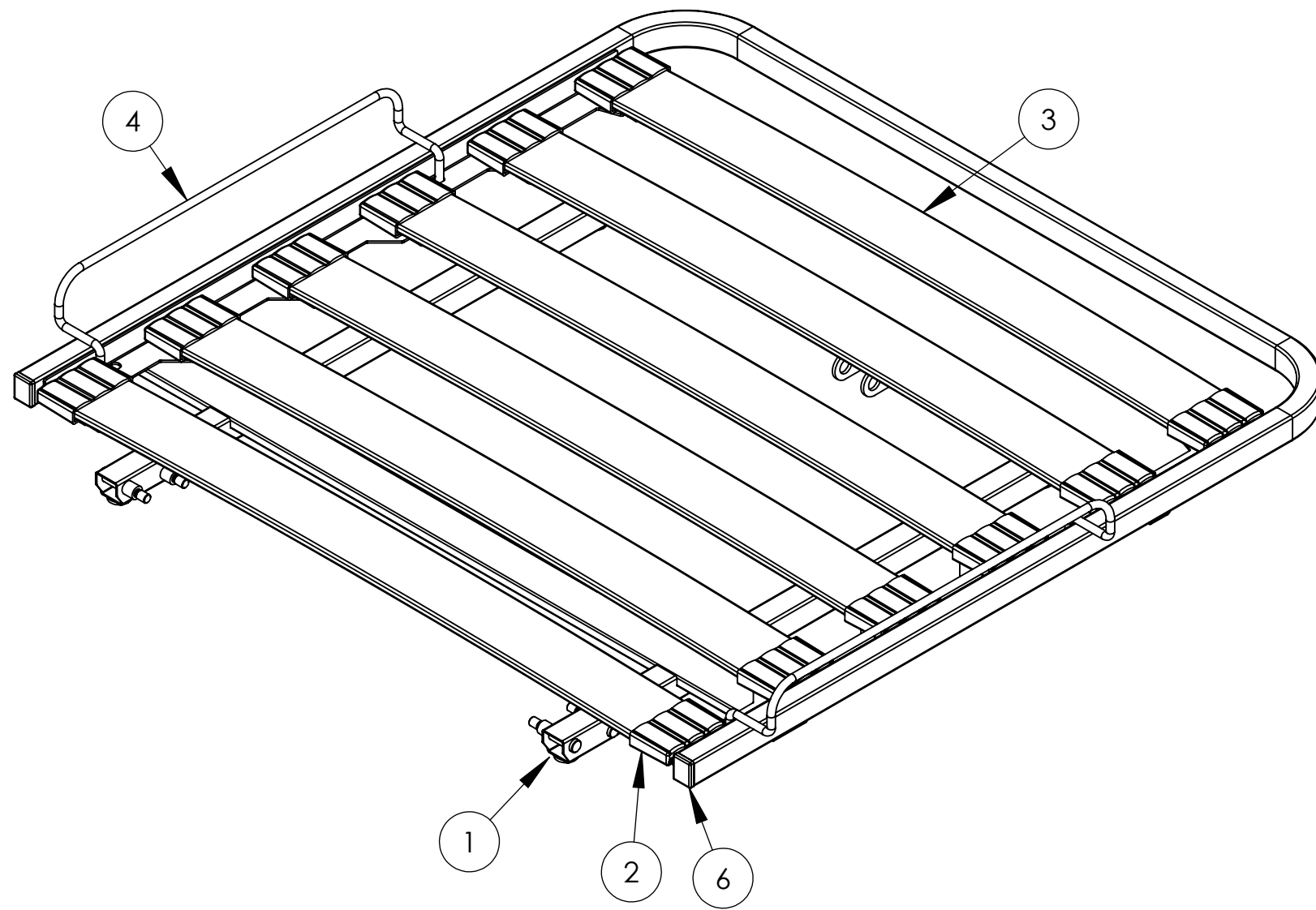


Nr.	Artikel Nr.	Benämning	Rygg/QTY.
1	CS154500	Aktiv Ryggdel 90	1
2	CS149800	Latthållare	12
3	CS149710	Latt 718	6
4	CS154099	Madrassbygel Rygg	2
5	AA201090	Filtrondell 20x2,5 Nr 16-20 vit	4
6	AA208072	Ändavslut 25x20 Svart	2

1 2 3 4 5 6 7 8

1 2 3 4 5 6 7 8

A
B
C
D
E
F

4X CS009001

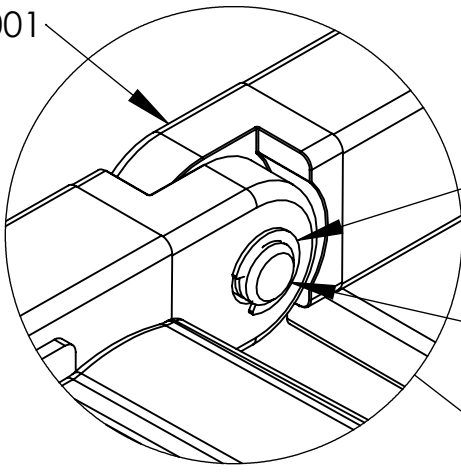

AA202024 2X

CS008003 2X

2X CS008012

CS007101

CS149710 7X

CS149800 13X

CS154711


AA202017 4X

CS034101


2X AA202129

This document is the property of Comfort System Scandinavia Aktiebolag and may not be copied without our written permission, and the contents thereof must not be imparted to a third party nor be used for any unauthorized purpose. Contravention will be prosecuted.

1 2 3 4 5 6 7 8

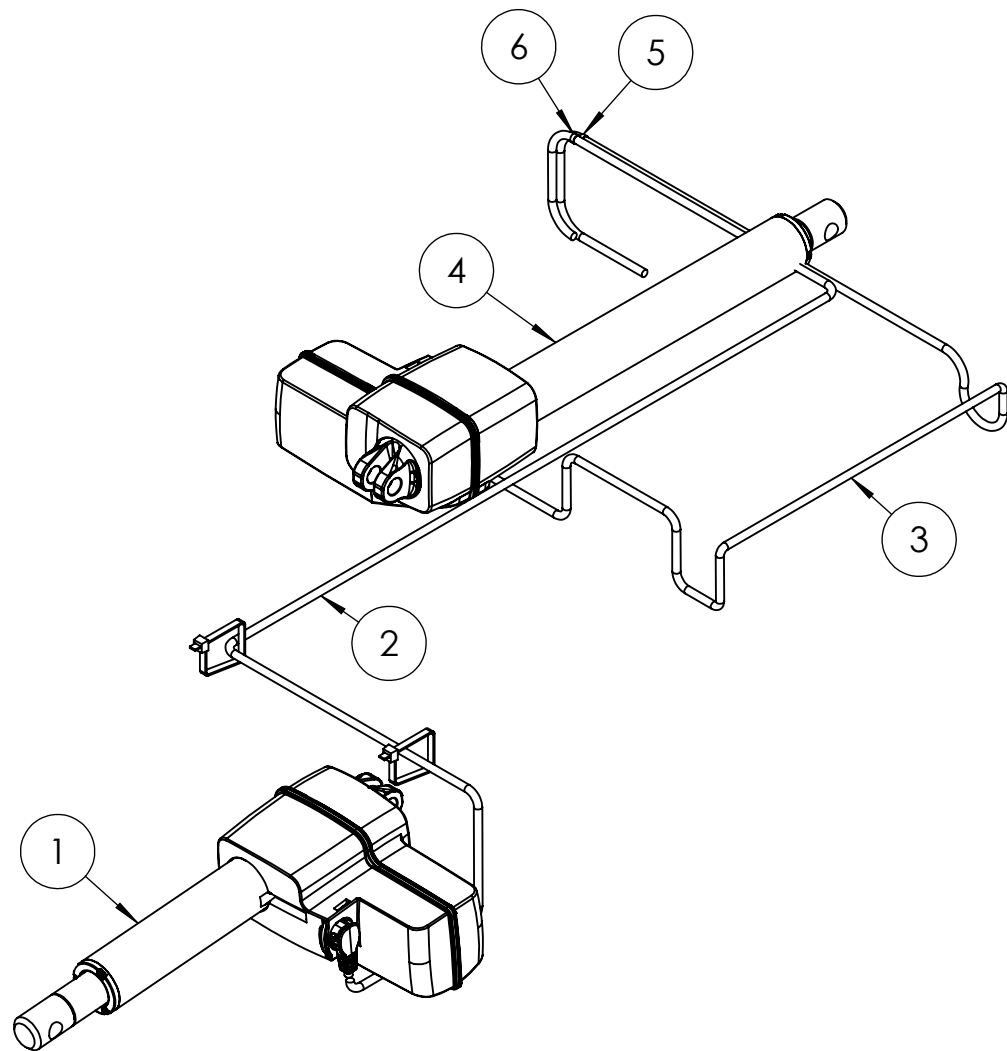


NR.	Artikel Nr.	Benämning	Motorsystem Sängbotten/QTY.
1	J14983	Benmotor LA40	1
2	00914657-1250-B	Motorkabel Fotmotor	1
3	00914681-1100	Motorkabel till ryggmotor	1
4	J14982	Ryggmotor LA40	1
5	AA-208080-1	Kabelnumrering #1	1
6	AA-208080-3	Kabelnumrering #3	1
7	AA208033	Buntband 4,8x200mm Natur	2


		◀	
 <p>Vistakullevägen 18 SE-561 46 Huskvarna Sweden Tel +46 (0)36 515 90 info@comfortsystem.se www.comfortsystem.se</p>		Titel / Benämning	
		Motorsystem Sängbotten CS4	
Artikel Nr		Rev	Blad
CS4B		1	1 (1)

This document is the property of Comfort System Scandinavia Aktiebolag and may not be copied without our written permission, and the contents thereof must not be imparted to a third party nor be used for any unauthorized purpose. Contravention will be prosecuted.


This document is the property of Comfort System Scandinavia Aktiebolag and may not be copied without our written permission, and the contents thereof must not be imparted to a third party nor be used for any unauthorized purpose. Contravention will be prosecuted.


Nr.	Artikel Nr.	Benämning	QTY.
1	CS514000	Saxram inre kpl. med snabbblås	1
2	CS510200	Chassiram C-lås delbar kpl	1
3	CS142100	Lyftram svetsad	1
4	CS143001	Saxram yttre svetsad	1
5	CS149211	Hjul C-låst 100x32 R-hjul lång tapp	1
6	CS149210	Hjul C-låst 100x32 lång tapp	3
7	CS148230	Länk C-låst Komplet	1
8	CS148221	Sexkantstång	1
9	CS148227	Sexkantstång	1
10	CS148801	Fotpedal Höger svetsad	1
11	CS148800	Fotpedal Vänster Svetsad	1
12	AA201002	Ändavslut 30x40x1,5-3	4
13	AA201001	Täckpropp 18 BX DP 687 Heyco	4
14	AA201004	Ändavslut Plastfot 60x40 svart	2
15	AA202096	Ändavslut 40x30x1-3	4
16	AA201223	Handtagsknopp Grön	2
17	AA201222	Handtagsknopp Röd	2
18	AA201219	Ändavslut IKPK 32-29 svart	4
19	AA201007	Täckpropp IKPT 16-14 svart	2
20	AA202013	BPF-F1G12 Låsstift	2
21	AA201070	Lagerbussning	4
22	CS149410	Spårkullager	4
23	AA202001	Glidbussning med fläns	2
24	AA202006	MC6S M10x30 fzb	4
25	AA202004	Spärring SGA 12	1
26	CS068905	CBH bult D12 L=50	1
27	AA201038	Distansbricka 25x13x7 svart	6
28	AA202007	K6S M8x12	4
29	AA202153	Låsbricka 8,4	4
30	AA202045	Stoppskruv M8x6	2
31	AA202341	Låssprint LCS rund 2,5 RF SS2331	1
32		Märketikett Sänglyft CS51.40H100	1
33	P51.002	Dekal Delbar sänglyft läs monteringsanvisning	1
34	P51.004	Dekal Lägesmarkering	1
35	P51.004	Dekal Lägesmarkering	1
36	P51.003	Dekal Låsbygel H-lyft	1
37		Etikett Serienummer	1
38	AA201011	Bricka Nylon 12,8x18x3	2
39	CS148041	Centrumaxel	1
40	CS148042	Gavelaxel	1
41	CS519114	Fästplåt elektronik	1
42	CS148032	Sprint	1
43	AA208078	Skruv MFT-TT M4x8	2

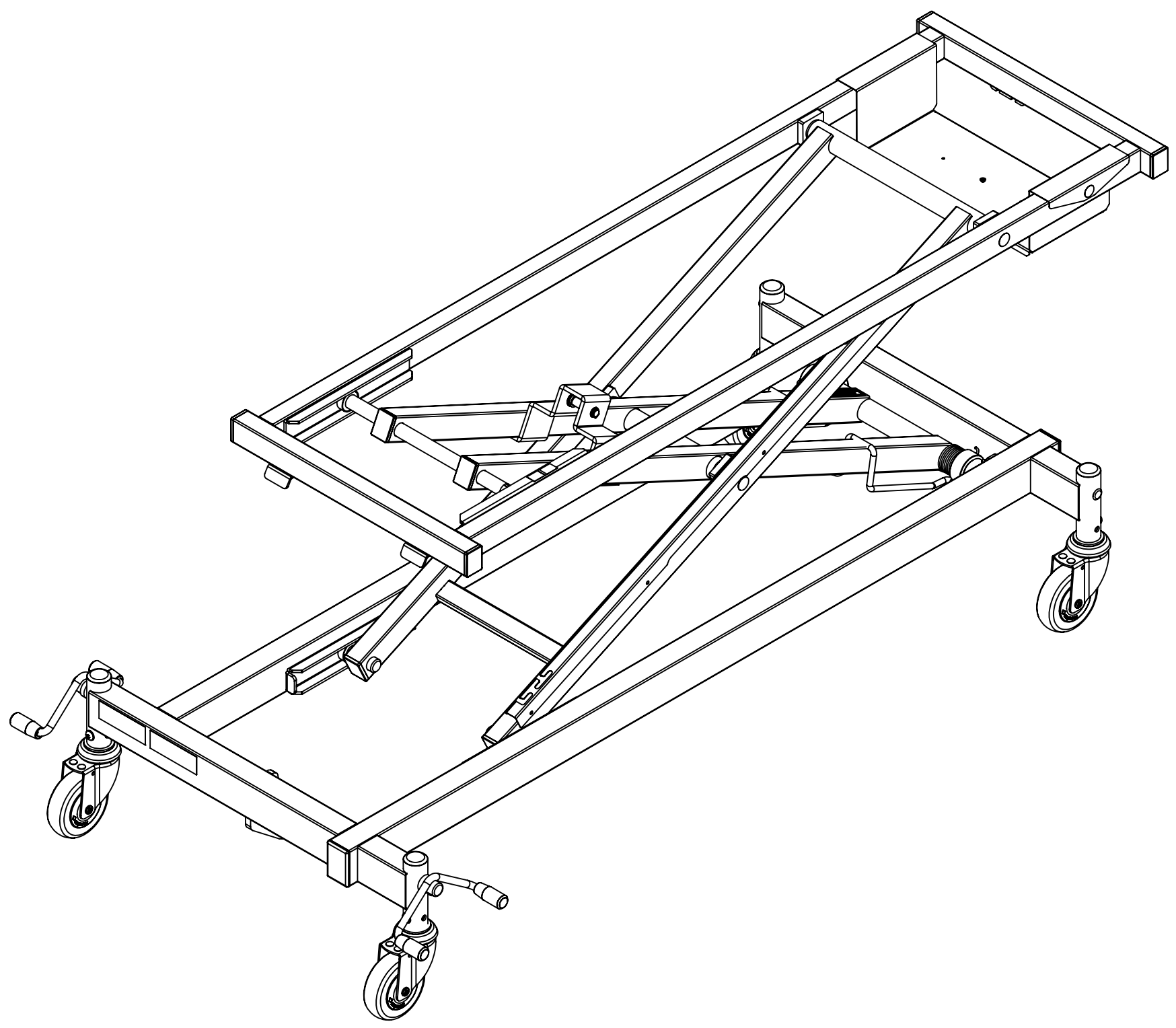


 <p>Vistakullevägen 18 SE-561 46 Huskvarna Sweden Tel +46 (0)36 515 90 info@comfortsystem.se www.comfortsystem.se</p>		Titel / Benämning	
		Sänglyft H100	
Artikel Nr		Rev	Blad
CS51.40H100		1	1 (6)


This document is the property of Comfort System Scandinavia Aktiebolag and may not be copied without our written permission, and the contents thereof must not be imparted to a third party nor be used for any unauthorized purpose. Contravention will be prosecuted.

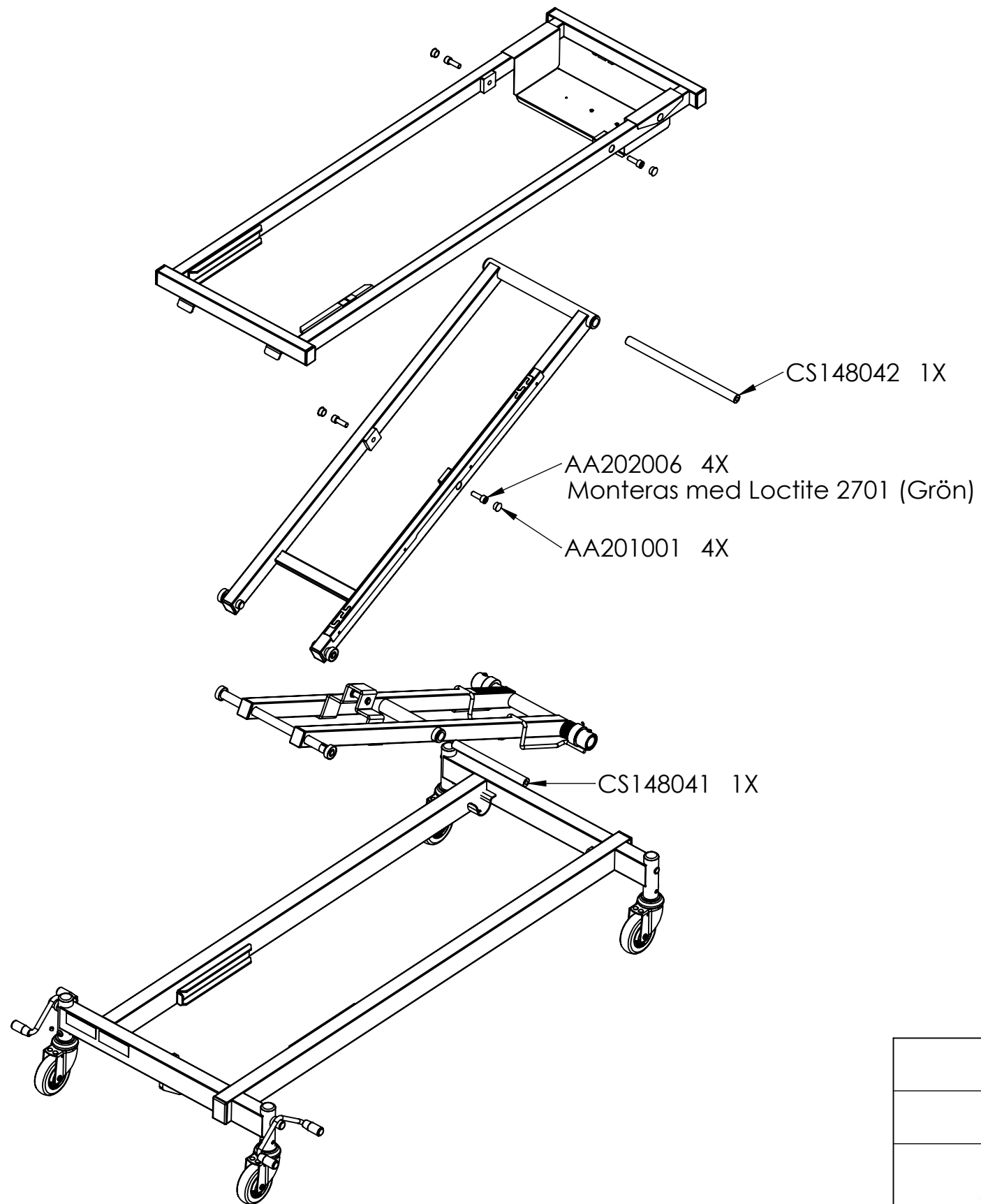




		◀	
 <p> Vistakullevägen 18 SE-561 46 Huskvarna Sweden Tel +46 (0)36 515 90 info@comfortsystem.se www.comfortsystem.se </p>		Titel / Benämning Sänglyft H100	
		Artikel Nr CS51.40H100	Rev 1


This document is the property of Comfort System Scandinavia Aktiebolag and may not be copied without our written permission, and the contents thereof must not be imparted to a third party nor be used for any unauthorized purpose. Contravention will be prosecuted.

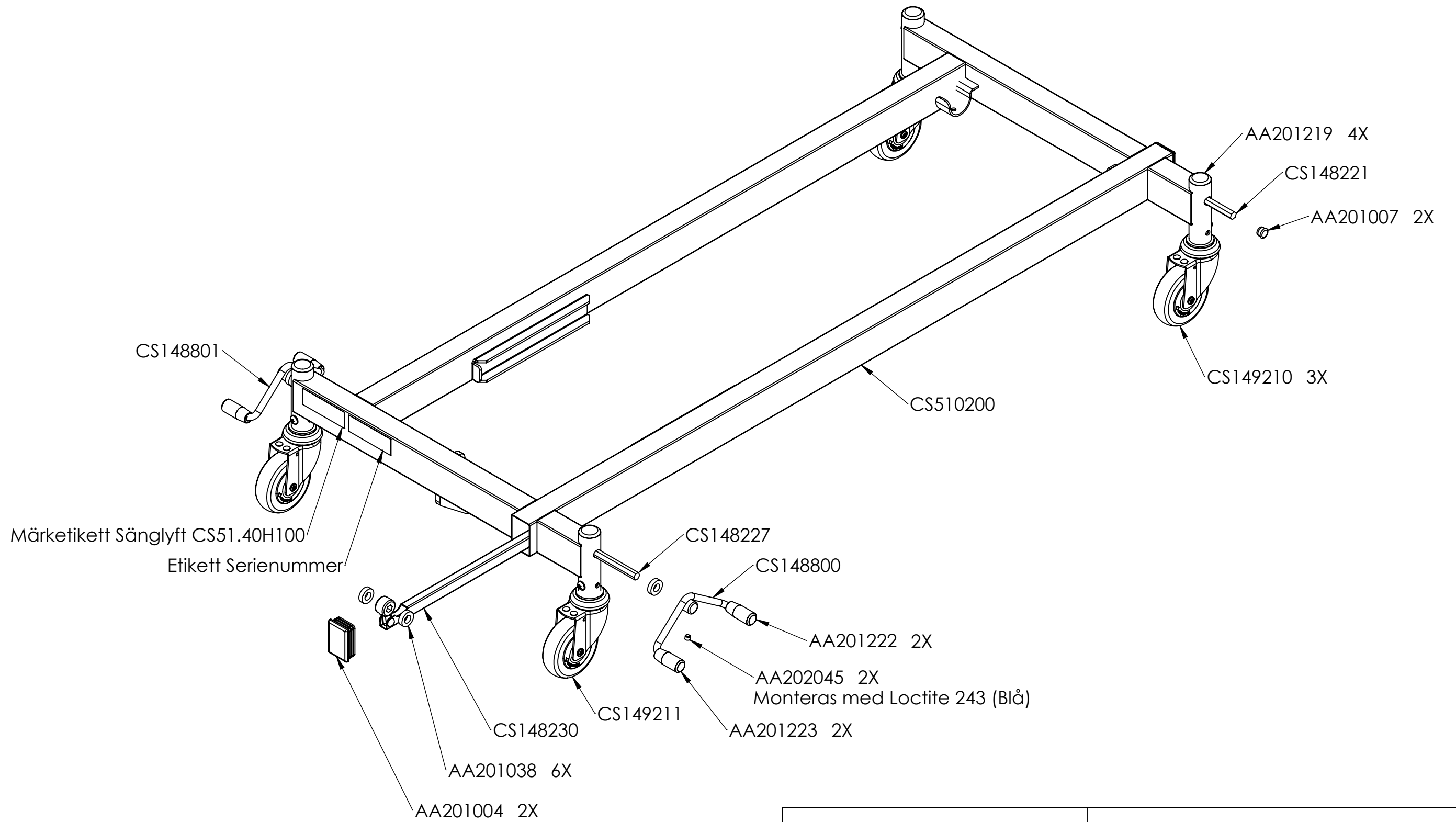


		◀	
 <p> Vistakullevägen 18 SE-561 46 Huskvarna Sweden Tel +46 (0)36 515 90 info@comfortsystem.se www.comfortsystem.se </p>		Titel / Benämning Sänglyft H100	
		Artikel Nr CS51.40H100	Rev 1


This document is the property of Comfort System Scandinavia Aktiebolag and may not be copied without our written permission, and the contents thereof must not be imparted to a third party nor be used for any unauthorized purpose. Contravention will be prosecuted.

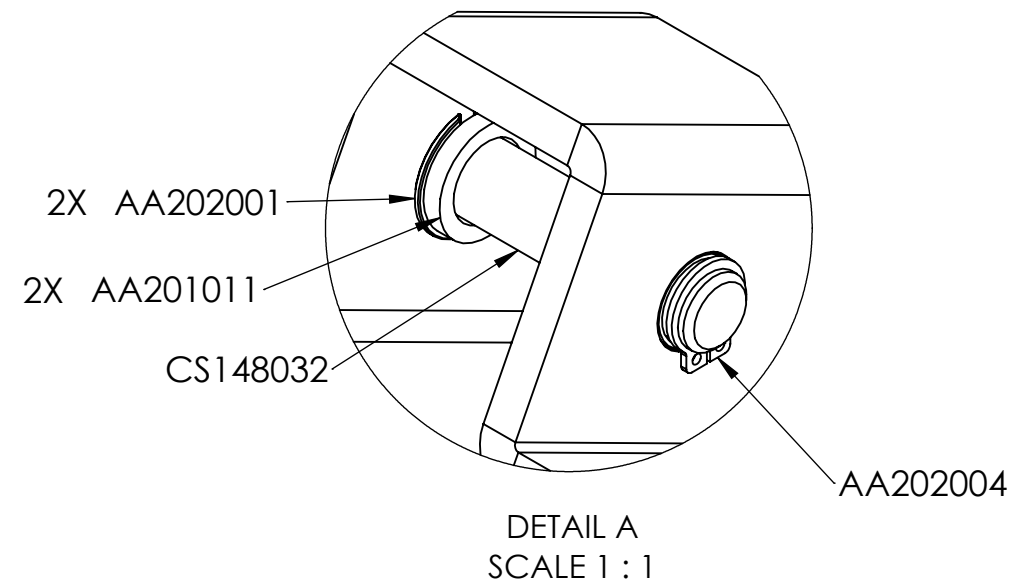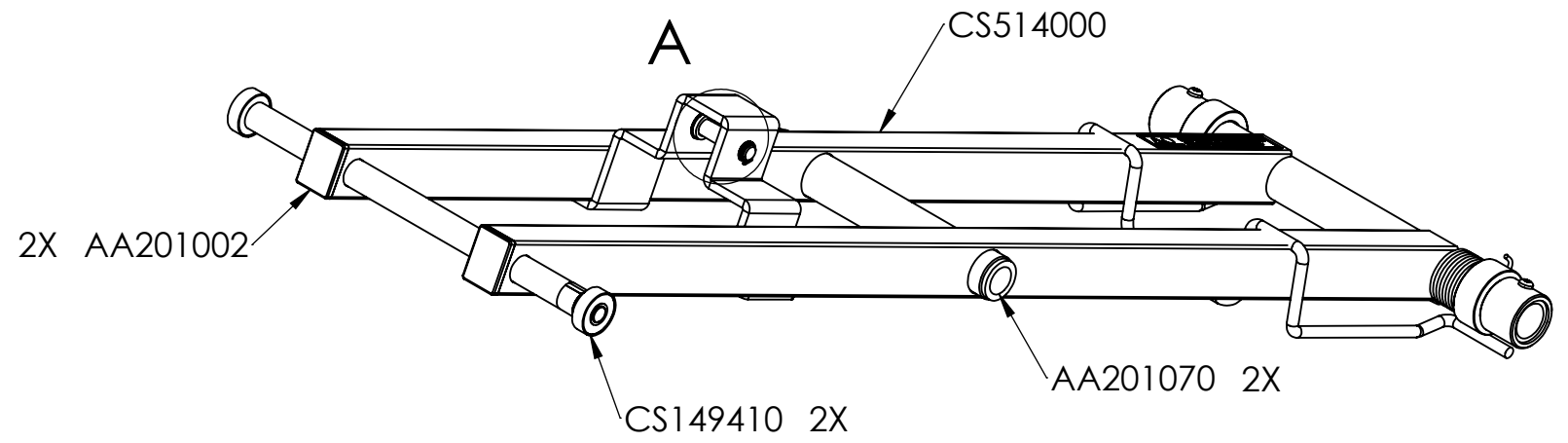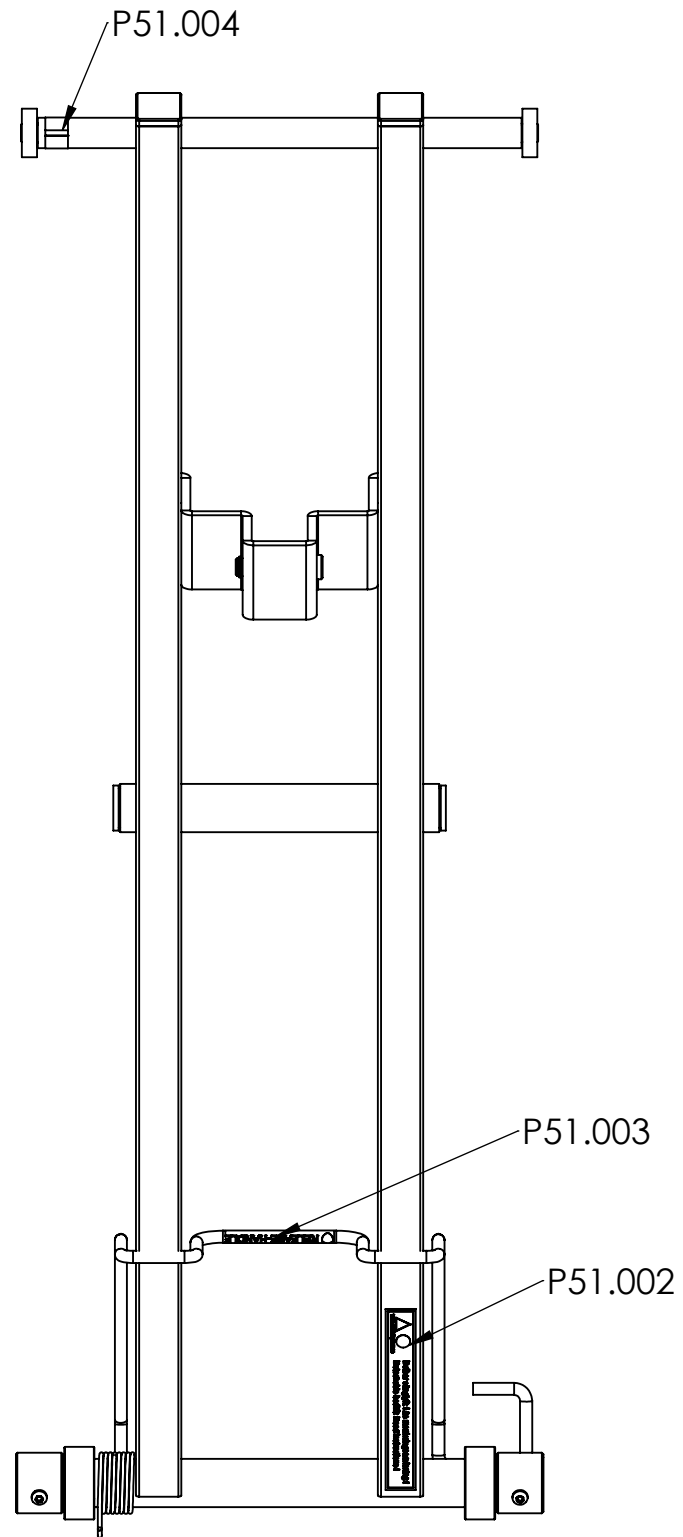


		◀	
 <p> Vistakullevägen 18 SE-561 46 Huskvarna Sweden Tel +46 (0)36 515 90 info@comfortsystem.se www.comfortsystem.se </p>		Titel / Benämning Sänglyft H100	
		Artikel Nr CS51.40H100	Rev 1


This document is the property of Comfort System Scandinavia Aktiebolag and may not be copied without our written permission, and the contents thereof must not be imparted to a third party nor be used for any unauthorized purpose. Contravention will be prosecuted.

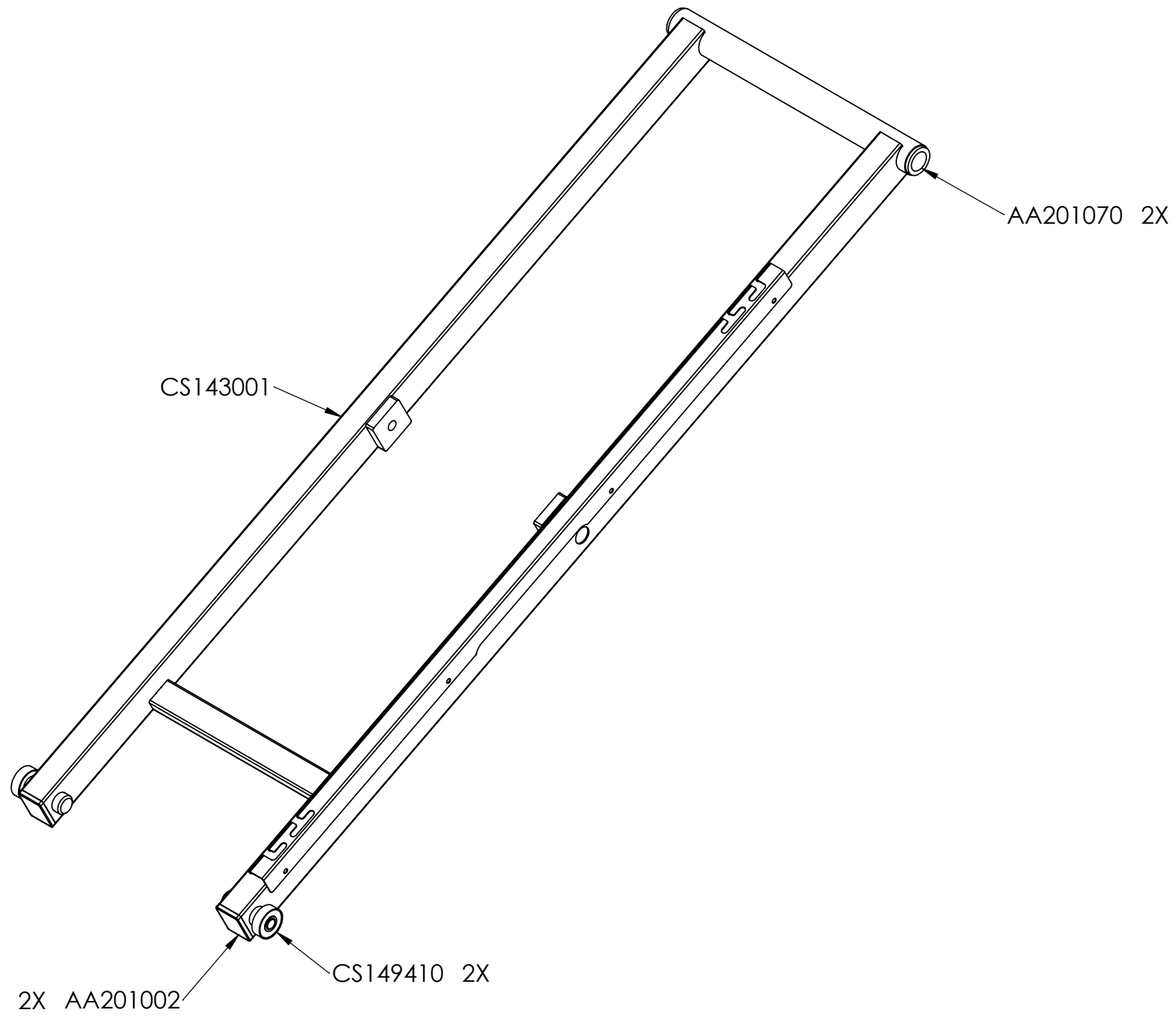


			◀
 Vistakullevägen 18 SE-561 46 Huskvarna Sweden Tel +46 (0)36 515 90 info@comfortsystem.se www.comfortsystem.se		Titel / Benämning Sänglyft H100	
		Artikel Nr CS51.40H100	Rev 1

This document is the property of Comfort System Scandinavia Aktiebolag and may not be copied without our written permission, and the contents thereof must not be imparted to a third party nor be used for any unauthorized purpose. Contravention will be prosecuted.

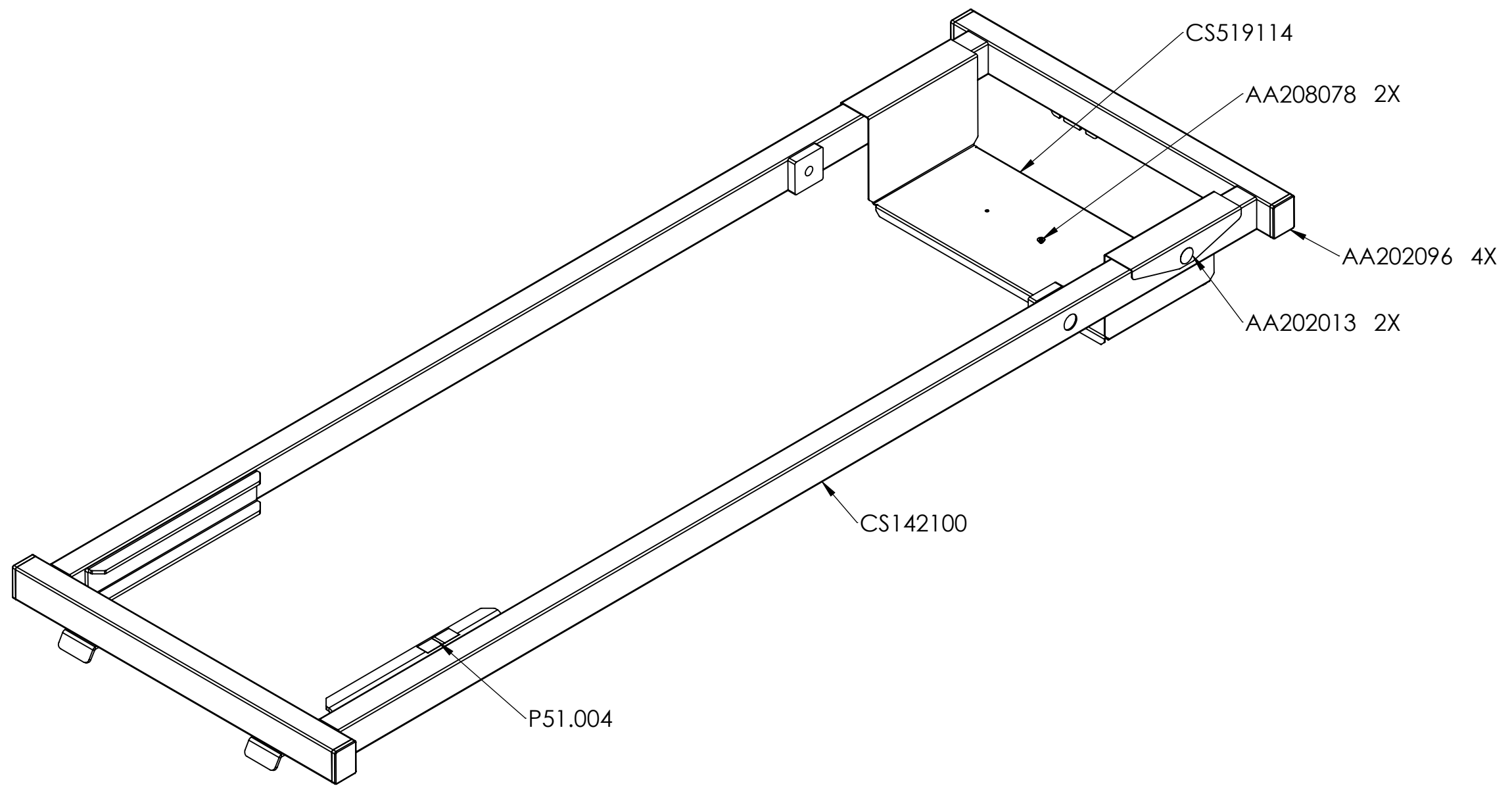


		◀	
 <p> Vistakullevägen 18 SE-561 46 Huskvarna Sweden Tel +46 (0)36 515 90 info@comfortsystem.se www.comfortsystem.se </p>		Titel / Benämning Sänglyft H100	
		Artikel Nr CS51.40H100	Rev 1

This document is the property of Comfort System Scandinavia Aktiebolag and may not be copied without our written permission, and the contents thereof must not be imparted to a third party nor be used for any unauthorized purpose. Contravention will be prosecuted.

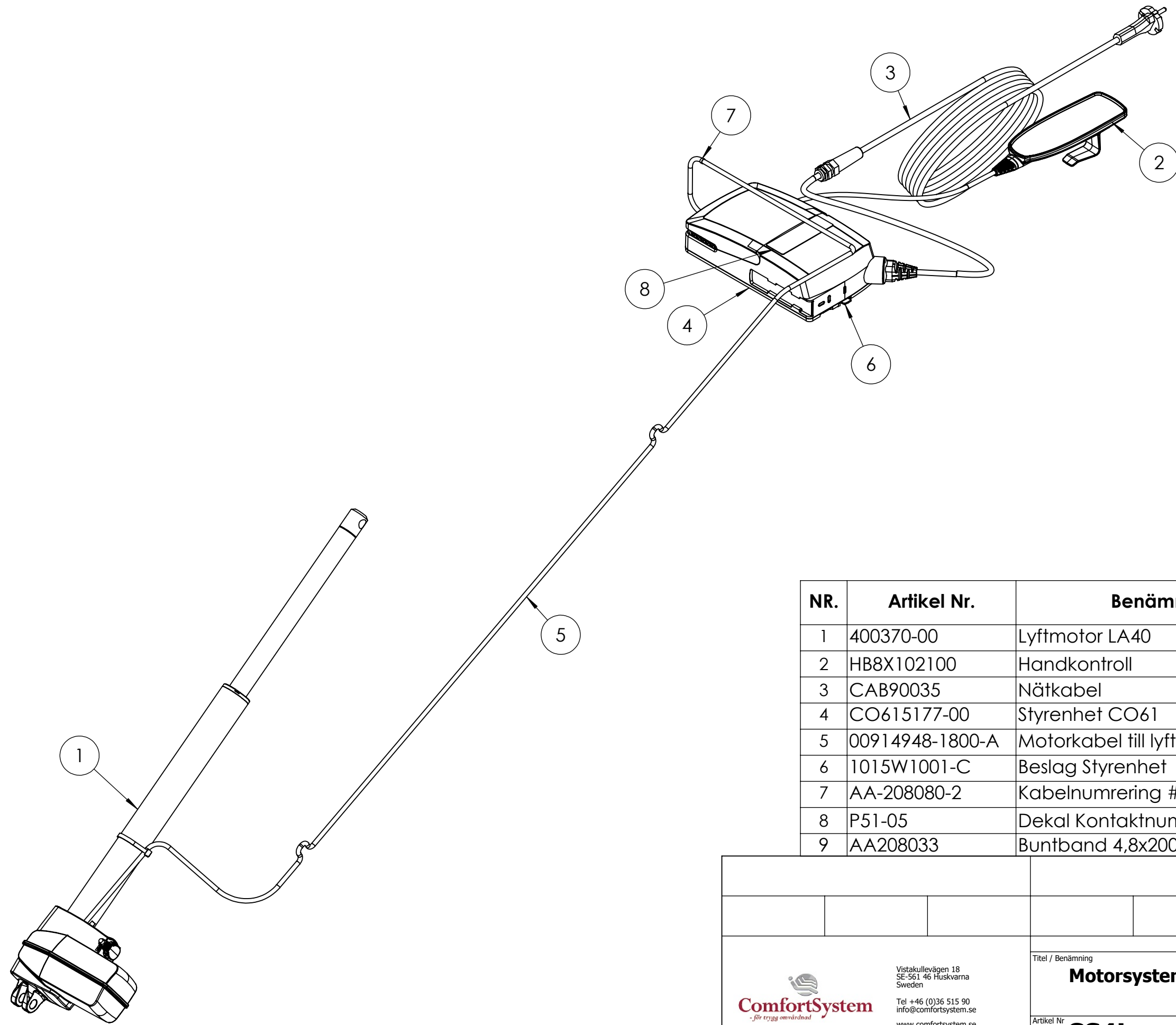

NR.	Artikel Nr.	Benämning	Motorsystem Sänglyft/QTY.
1	400370-00	Lyftmotor LA40	1
2	HB8X102100	Handkontroll	1
3	CAB90035	Nätkabel	1
4	CO615177-00	Styrenhet CO61	1
5	00914948-1800-A	Motorkabel till lyftmotor	1
6	1015W1001-C	Beslag Styrenhet	1
7	AA-208080-2	Kabelnumrering #2	1
8	P51-05	Dekal Kontakt nummer	1
9	AA208033	Buntband 4,8x200mm Natur	1

 <p>Vistakullevägen 18 SE-561 46 Huskvarna Sweden Tel +46 (0)36 515 90 info@comfortsystem.se www.comfortsystem.se</p>		Titel / Benämning	
		Motorsystem Sänglyft CS4	
Artikel Nr		Rev	Blad
CS4L		1	1 (1)