
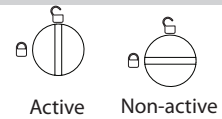


Artena Hospital

Manipulation	Hand switch	Hand switch HD	Lock box ACL
BACKREST 			
KNEE REST 			
RELAXPOSITION 			
HIGH/LOW 			
SLEEPINGPOSITION 			

Lock/ Unlock

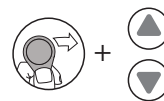

Trend and Anti-trend
Not possible for
Artena


The available bed functionality is in line with the chosen and ordered configuration.






Artena Hospital


Quick release

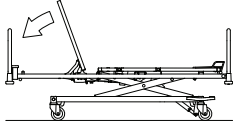

Lever

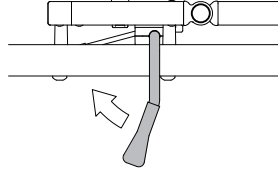

Bed extension

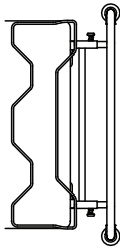




1. lift the footrest
2. pull out both poll knobs
3. pull out the bed extension
4. lock the buttons
5. pull out the foot extension

Brake mechanism

1 directional castor, fixed

4 castors unbraked

4 castors braked

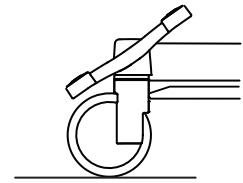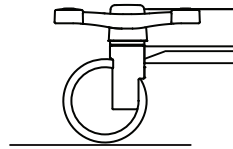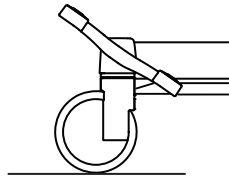

The available bed functionality is in line with the chosen and ordered configuration.

