
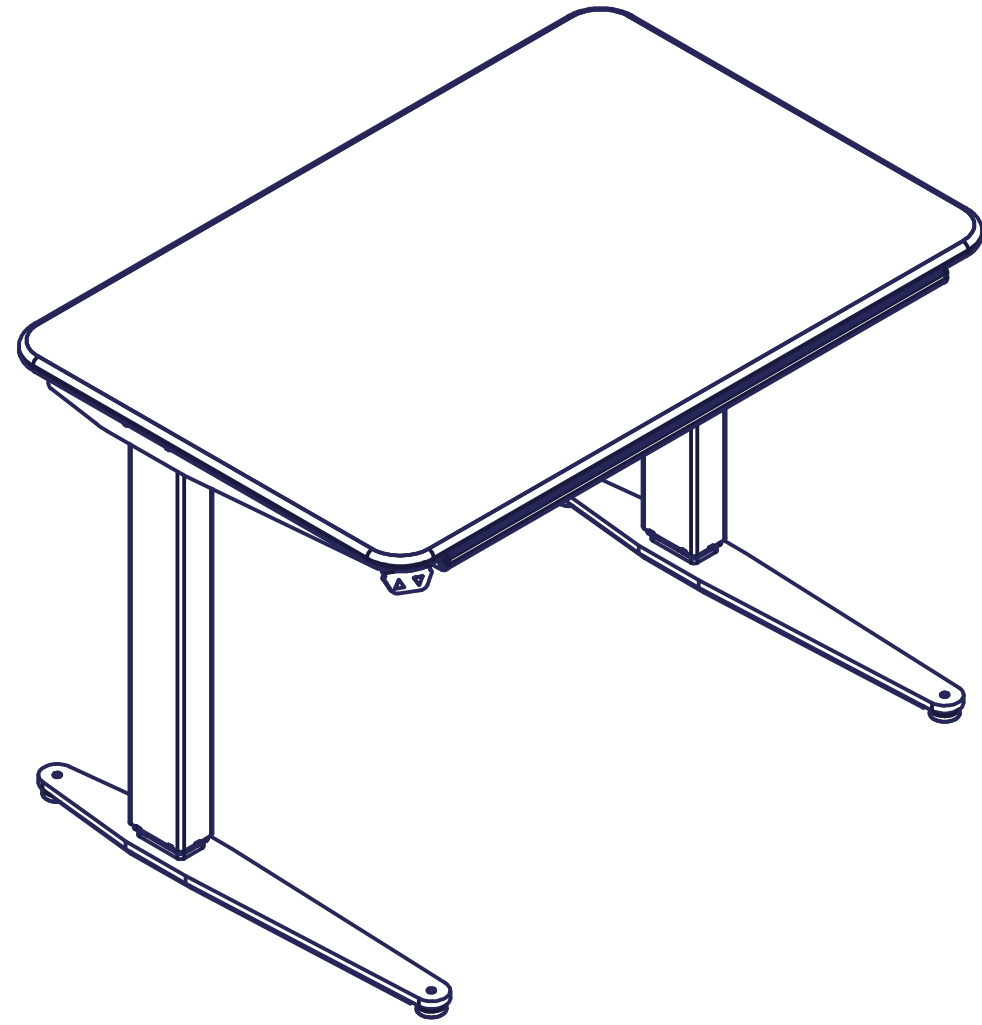
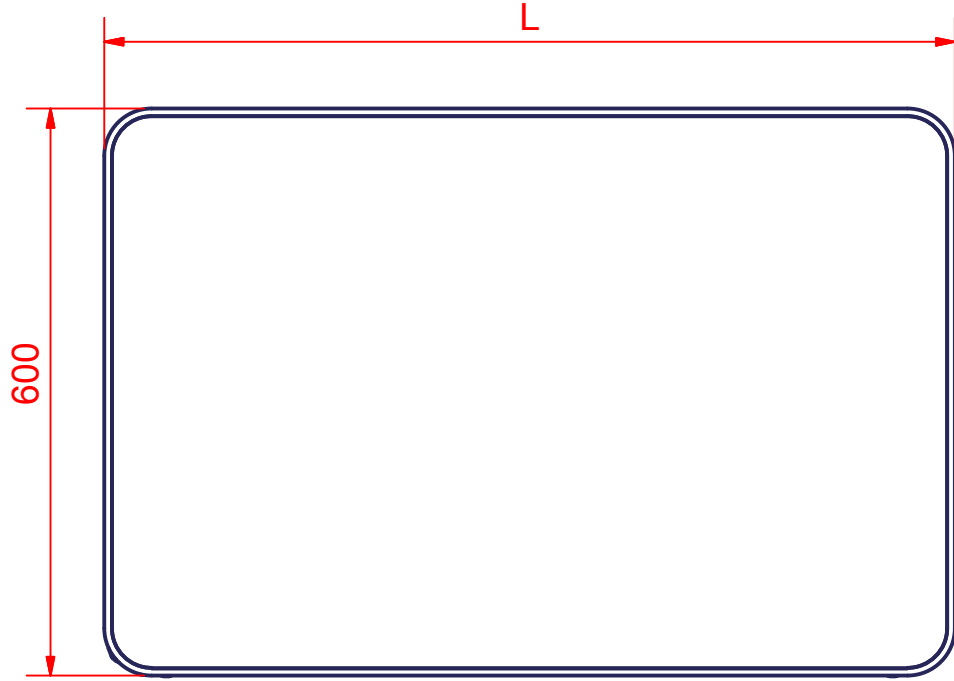


Number	Lenght(L)	Color
20-60800-2	90cm	Lightgrey
20-60810-2	120cm	Lightgrey

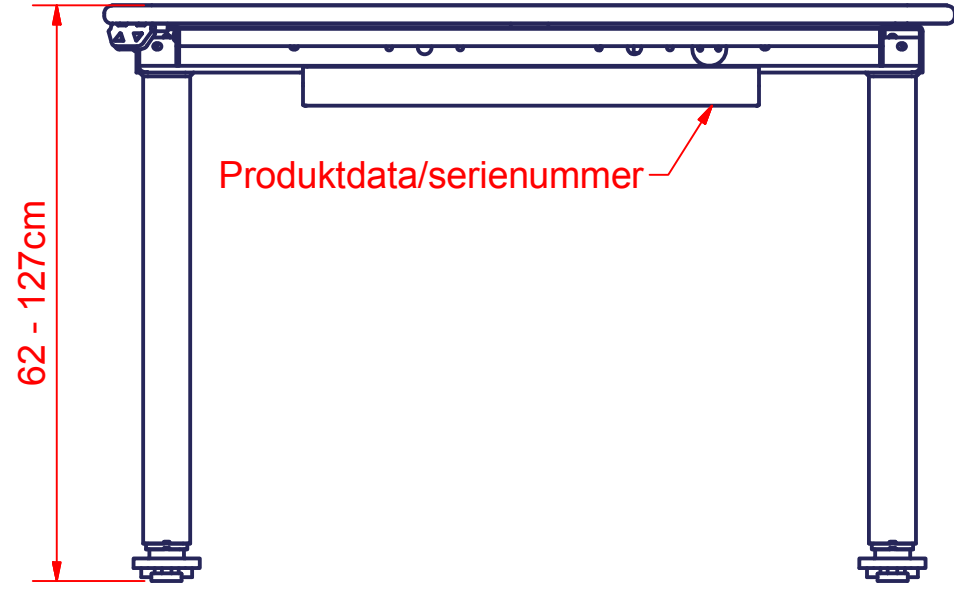

High/low A 90/120cm Reservedele	
------------------------------------	--

Tegn.	sj	15-09-2014
DS/ISO 2768 m		
Vægt:	N/A	N/A

ROPOX Better ways to better days
 Ringstedgade 221, 4700 Næstved
 Tlf.: 55750500 Fax.: 55750550


Tegning nr.:	20-60800-2
--------------	------------



90x60cm			
ITEM	QTY	TITLE	NUMBER
9	1	Bordplade	64109060
10	1	Overramme	20-60800-084
11	1	Frontprofil	20-60320-077
12	1	Bundlæg	20-60800-095

120x60cm			
ITEM	QTY	TITLE	NUMBER
9	1	Bordplade	64112060
10	1	Overramme	20-60800-101
11	1	Frontprofil	20-60320-108
12	1	Bundlæg	20-60800-105

95010011	2				15
95010008	2				14
93000006	1	Netkabel			8
93000068	2	Motorkabel 100cm			7
93000039	1	Håndkontakt			6
97702-034	1	Bordpladebeslag			5
96100916	4	Stilleskrue M10x30			4
93000179	1	Kontrolboks CBD6S			3
93000175	2	Motor DL16			2
20-60800-078	2	Fod til bord			1
PART NUMBER	QTY	TITLE	L	B	ITEM

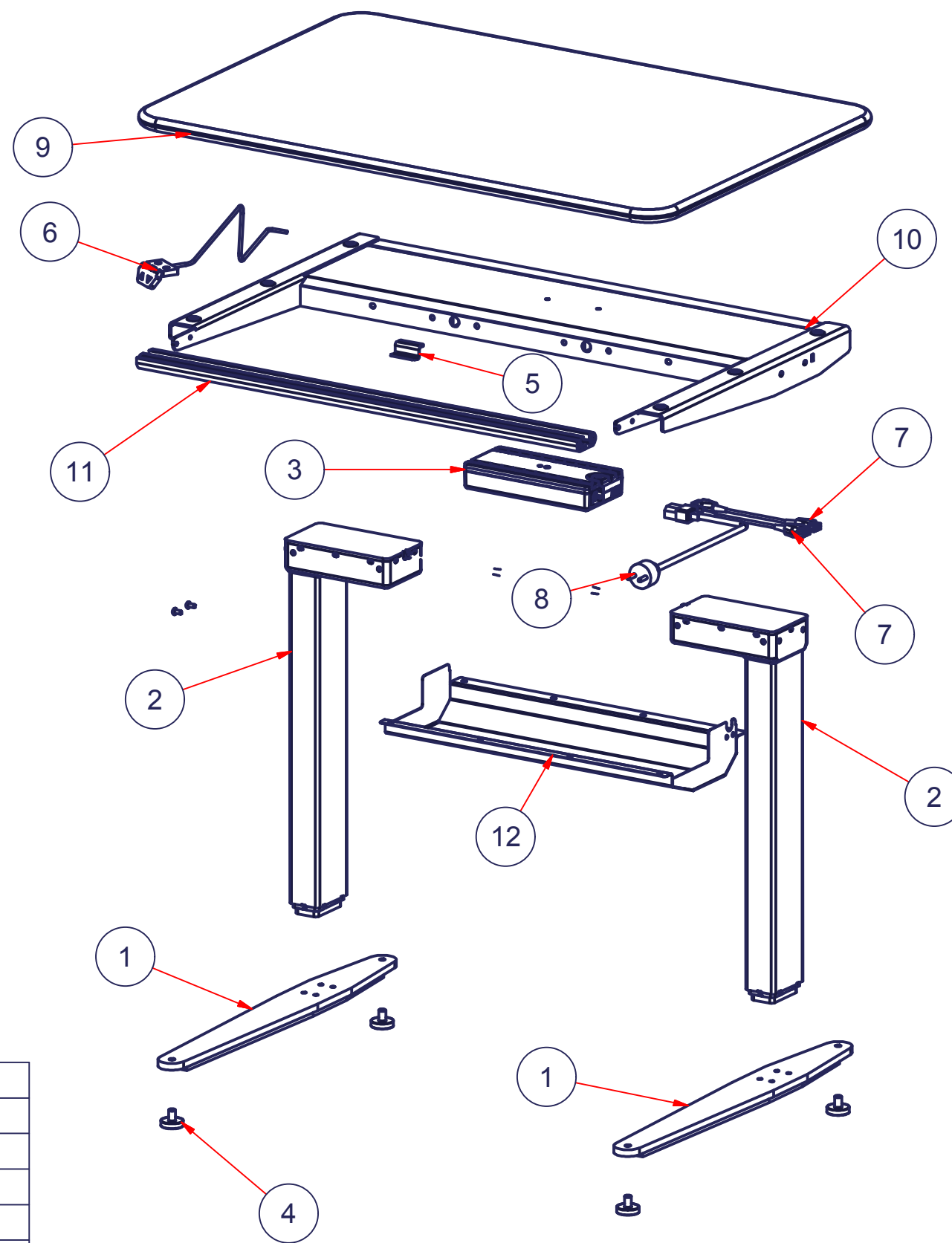

Side 2 af 2		High/low A 90/120cm Reserve dele		Tegn. sj	15-09-2014
				DS/ISO 2768 m	
		Vægt:	N/A	N/A	
		Tegning nr.:		20-60800-2	
		ROPOX Better ways to better days		Ringstedgade 221, 4700 Næstved Tlf.: 55750500 Fax.: 55750550	