Ropox Puslebord Medi 2

Brugermanual

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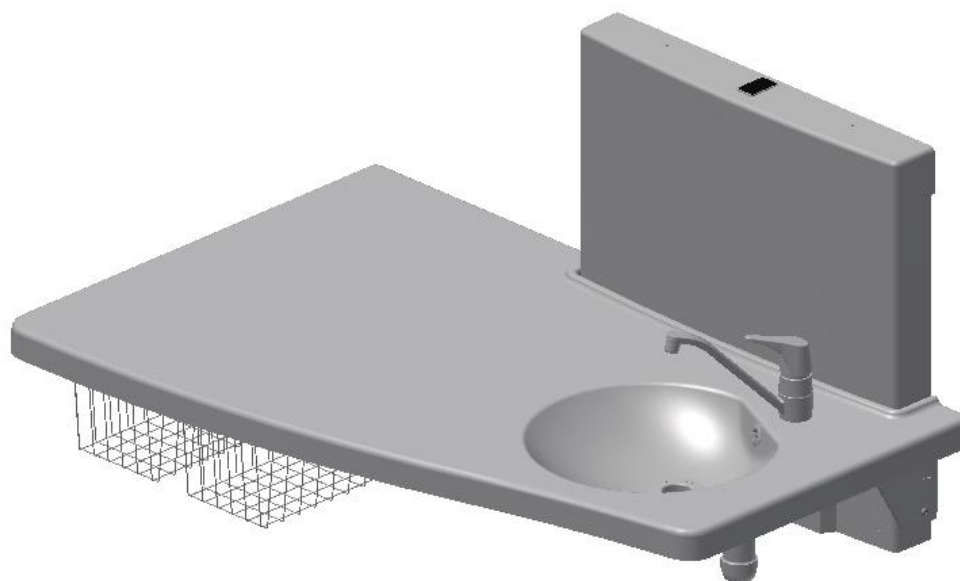
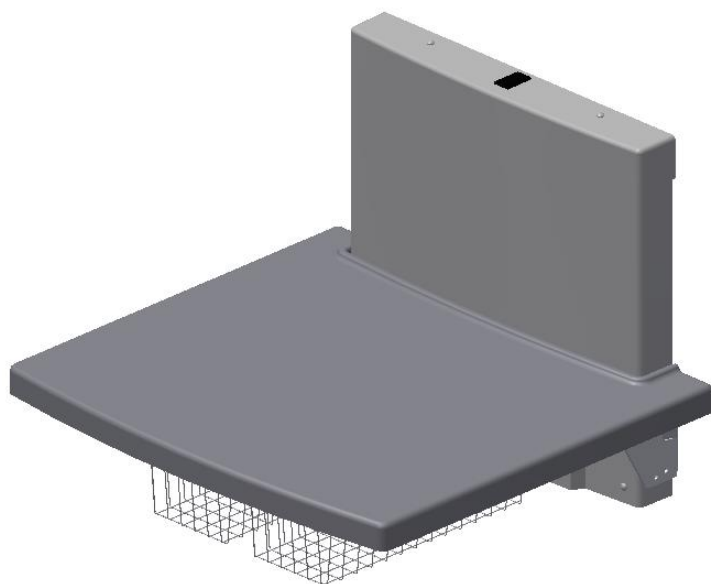
User Manual

Mounting Instructions

Keep this manual with the product at all times

Contents

Product information	25
Item numbers	25
Technical data	26
Daily use	26
Recommendations for use	27
Important information prior to mounting	28
1. Wall mounting	30
2. Mounting of cover	32
3. Mounting of tabletop	33
4. Connection of water and drain	34
Wiring diagram	36
Connections	37
Dismounting	37
Marking	38
Label	38
Symbols	38
Support	39
Fault finding	39
Waste handling	39
Factors affecting the product during storage and transport	39
Maintenance	40
Spare parts	41
Maintenance	42
Cleaning	42
CE Declaration	43
EU – Declaration of Conformity	43
Complaints	44

General information

Product information

It is important to read this manual prior to mounting and use.

Product : Ropox Nursing Table Medi 2


Manufacturer: Ropox, DK-4700 Næstved, Tel. +45 55 75 05 00

Application – The height-adjustable nursing table is designed for nursing persons up to 75 kg. The nursing table may be used in private homes, care centres or the like to ensure correct and optimum working conditions for the staff/user.

The table is adjustable in height by pressing the UP and DOWN buttons of the hand-control switch.

Item numbers

Length of nursing table	Item number
90 cm	50-50677
130 cm	50-50675

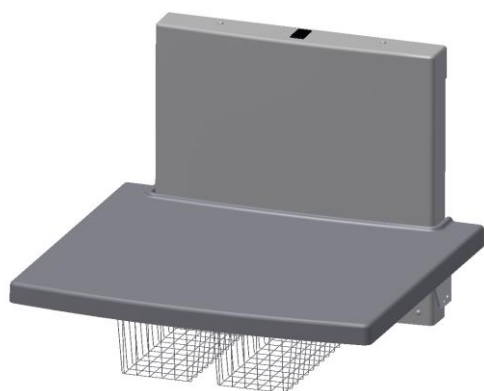Nursing table Medi 2 – 90 cm

Nursing table Medi 2 – 130 cm

General information

Technical data

Power supply:	230V, 50Hz, max 2.0A
Standby consumption:	7W
Duty cycle:	Max. 10%, 2min/18min
Height:	Working height 65 – 100cm Unit W=70cm, H=102cm, D=10/12,4cm
Material – frame:	Welded steel pipe St.37, stainless and aluminium
Tabletop:	ABS / PMMA
Surface treatment:	Powder-coated, white RAL 9010 mat, chromate
Max. load:	75 kg

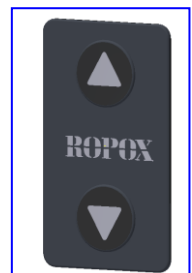
This product has CE marking according to the Machinery, Low Voltage and EMC Directives. It has been tested according to the European standards DS/EN ISO 12182 and DS/EN ISO 60335.

Daily use

Before using this product, connect the mains voltage as described under connections.

The height of the nursing table is adjustable by pressing the UP and DOWN buttons of the hand-control switch.

Max. load of nursing table: 75kg.

Warning !!

During height adjustment of the nursing table, there is a risk of getting jammed between table and adjacent objects.

Before using the table, ensure that there are no objects preventing movement of the table.

!! The product must NOT be operated by children!!

General information

Recommendations for use

The height adjustment allows individual working height for each carer and contributes to reducing the need for heavy lifting.

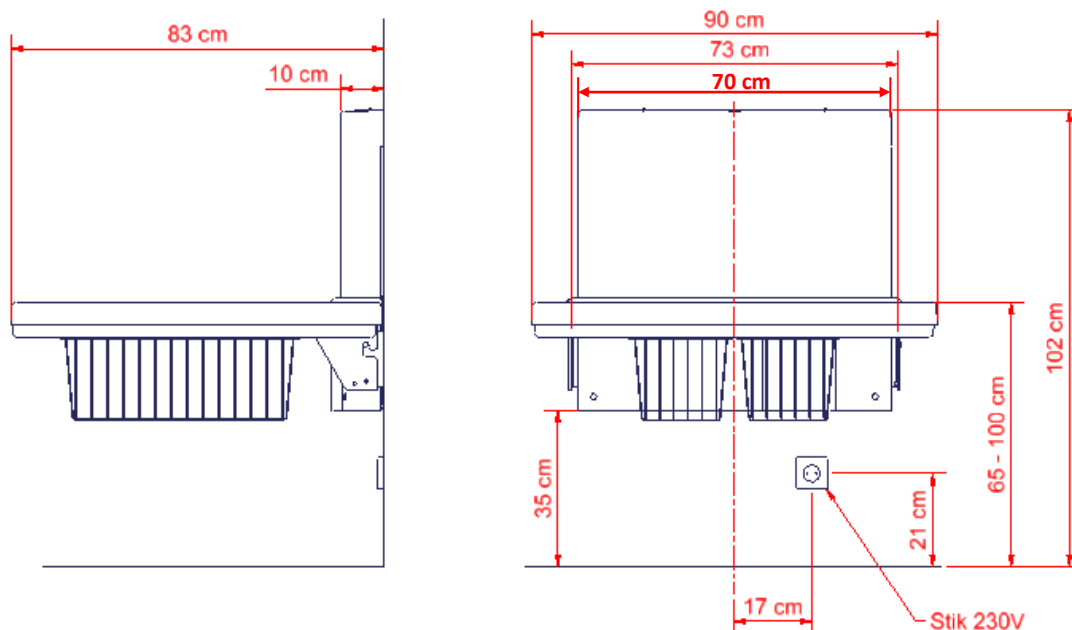
To ensure optimum utilisation of the functions of the nursing table and to avoid accidents it is important to observe these basic safety procedures.

- NEVER leave a person on the nursing table without supervision – not even for a short period of time.
- The nursing table should only be operated by adults.
- Pay special attention to persons in the room during use of the nursing table. Make sure that there are no children playing under or with the nursing table.
- Make sure that children do not operate the nursing table unintentionally.
- See that there is free space above and below the nursing table to allow height adjustment. Make sure that there are no obstacles above, below or around the nursing table in order to avoid the risk of getting jammed during height adjustment.
- If the hand-control switch is damaged or out of order, it must be replaced immediately. The buttons are of the “press and keep pressed” type. The motor stops when you let go of the button. This is an important safety factor.
- If two nursing tables have been mounted in the immediate vicinity of each other, it is recommended to fit a side cover between them to avoid the risk of getting jammed.
- During mounting, consider the jamming risk.


Mounting instructions

Important information prior to mounting

Dimension sketch of nursing table Medi 2:

Nursing table Medi 2, 90 cm

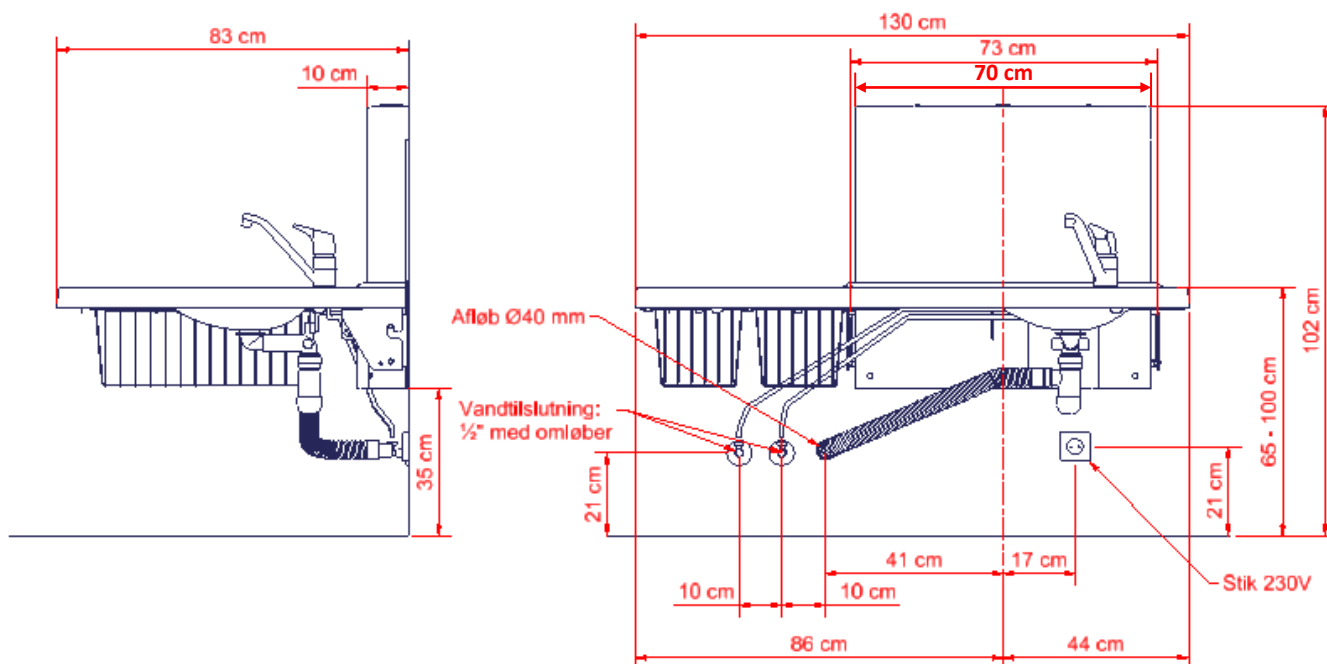
Nursing table Medi 2, 130 cm

Mounting instructions

These mounting instructions comprise:

Lifting unit	x1	Rawplugs	x7
Cover for lifting unit	x1	Bolt M5 x 25	x6
Tabletop	x1	Bolt M4 x 10	x9
Drain hose	x1	Bolt M8 x 20	x2
Assembly bolts	x7		

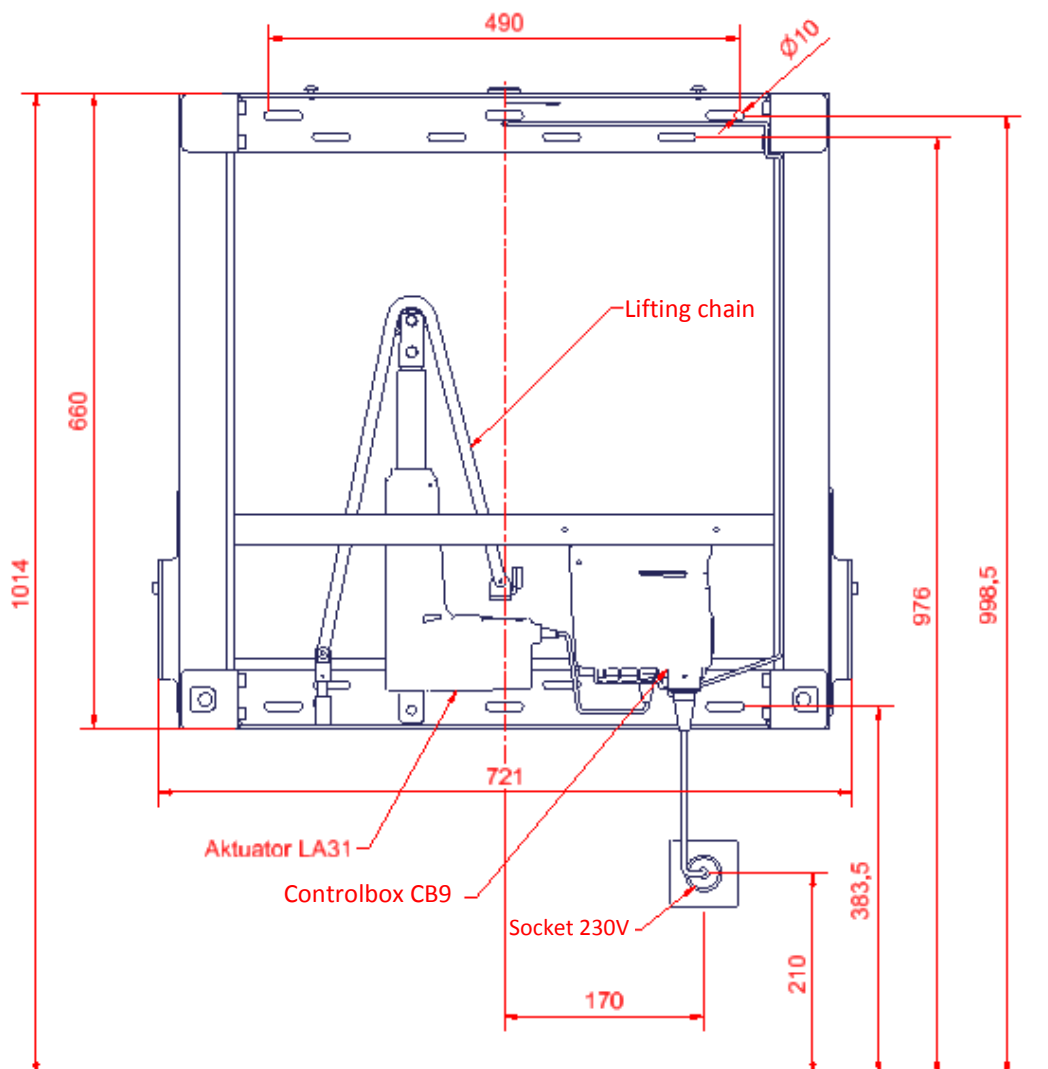Mounting Instructions

1. Mounting to the wall

The nursing table is being supplied with a set (8 pcs) of screws and rawlplugs, which is ONLY used to mount the unit in a concrete wall. The installer must always examine the material, condition and strength of the wall and use appropriate screws and rawlplugs.

Nursing table has a max user weight of 75 kg.

Maximum load in the top screws are 26kg for each screw.

If you have other objects mounted right next to this nursing table, you have to consider the risk of trapping, and therefore consider a safe distance to other objects.

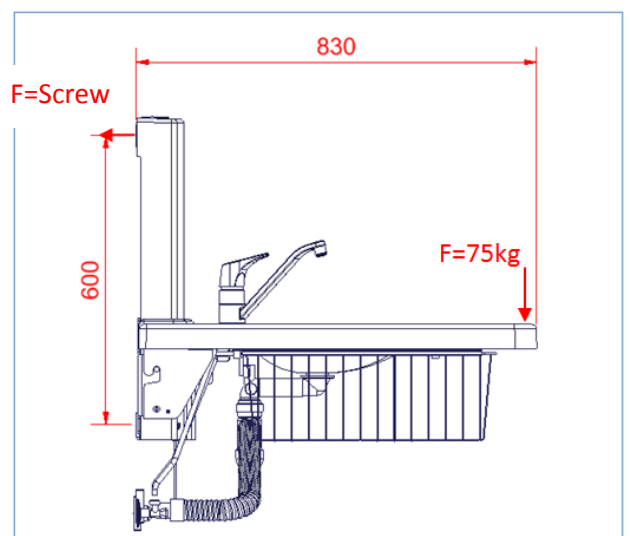
The holes for fastening can be marked up on the wall, four possibilities in the top, and four in the bottom of the frame. Place the frame against the wall and make sure it is horizontal. The frame is standing on two adjustable feet. The holes in the frame are all \varnothing 10 mm. Remove the frame and drill the holes.

Make sure that no dust or other obstacles are interfering with the rollers, chain, sprocket or gas springs. This can cause malfunction and damage the system

Force for each of the four screws in top of the unit:




$$F_{\text{screw}} = (75 \text{ kg} * 830) / (790 * 4 \text{ screws})$$

$$F_{\text{screw}} = 26 \text{ kg for each screw}$$


Mounting instructions

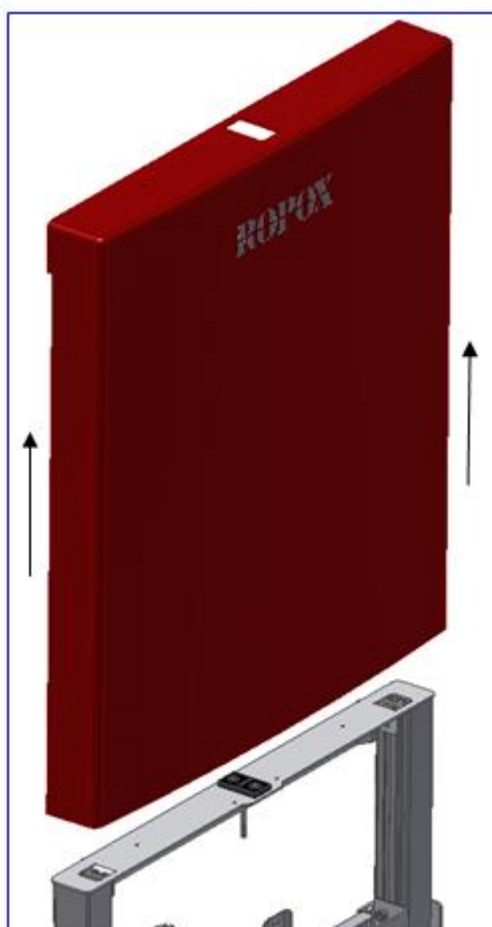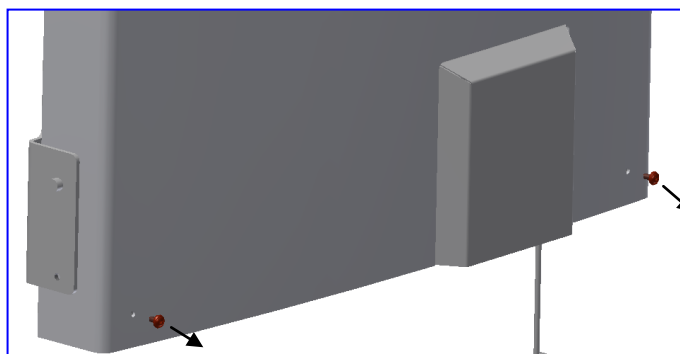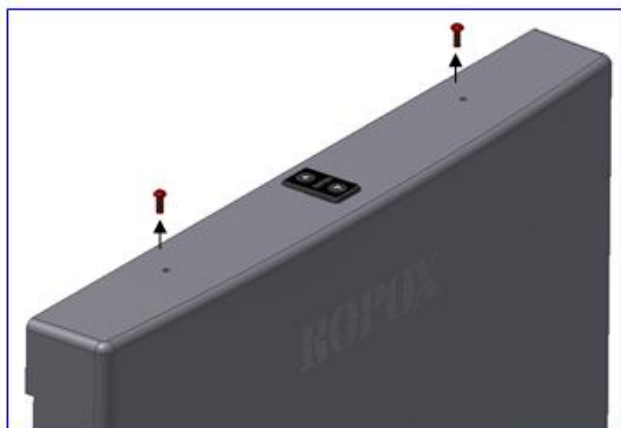
Wall mounting


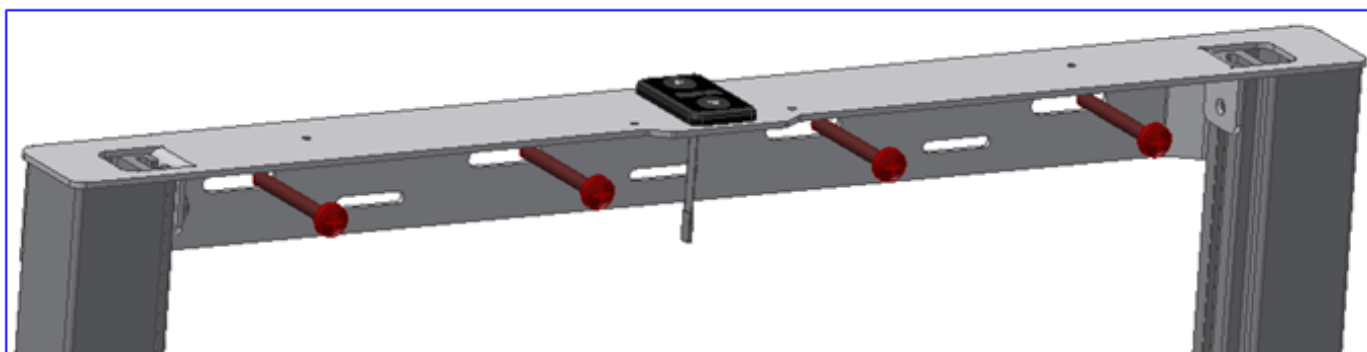
!! If the nursing table is to be provided with a leg section, follow separate mounting instructions for leg section before fastening the lifting unit on the wall !!

Mounting instructions

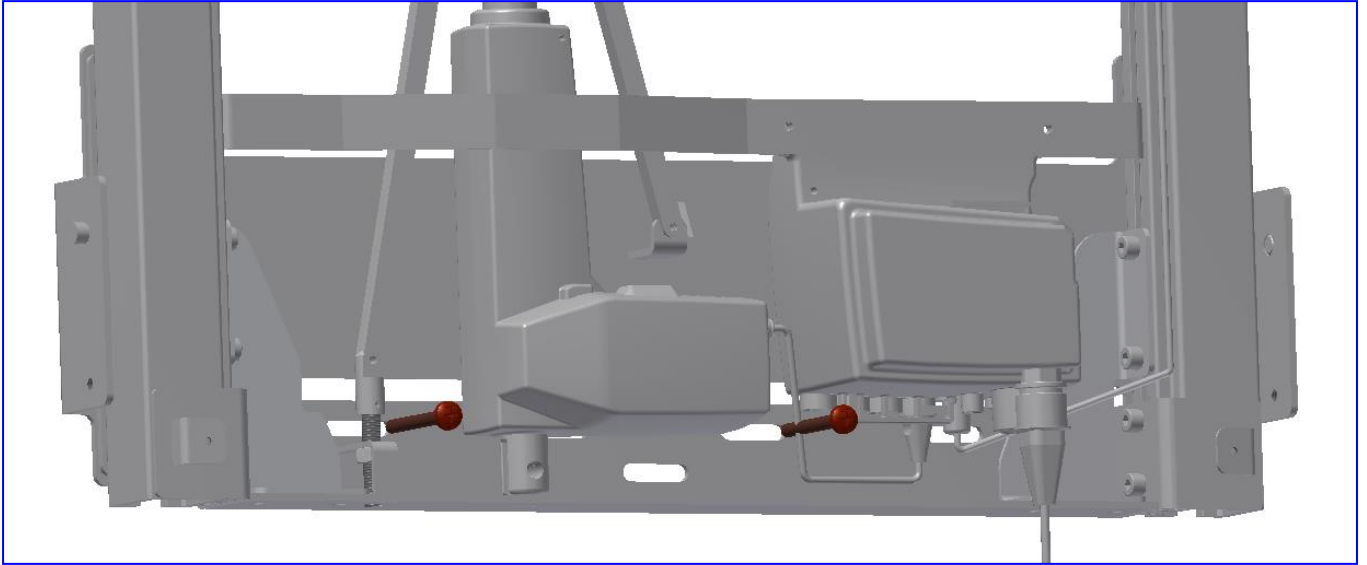
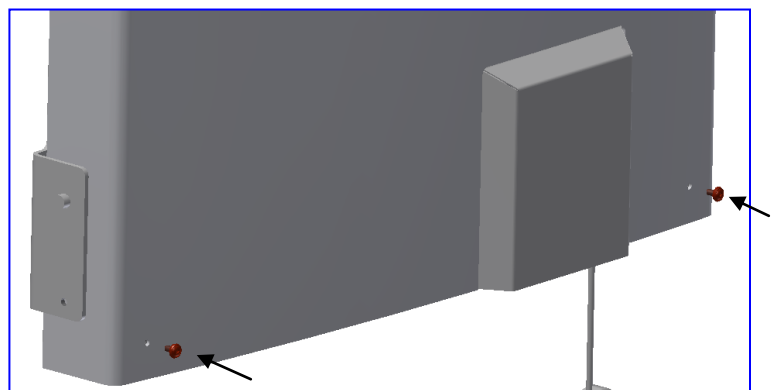
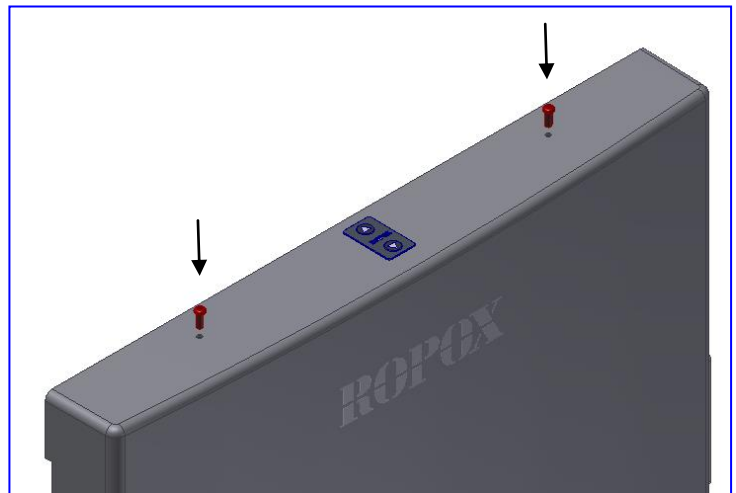
2. Mounting of cover




Mounting instructions

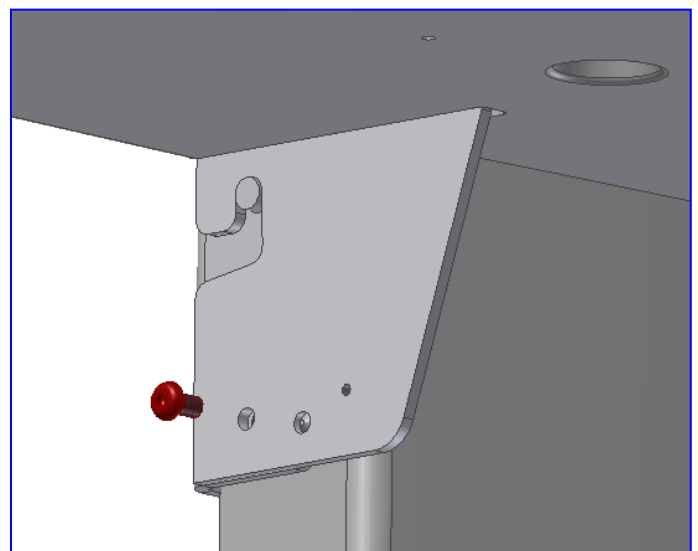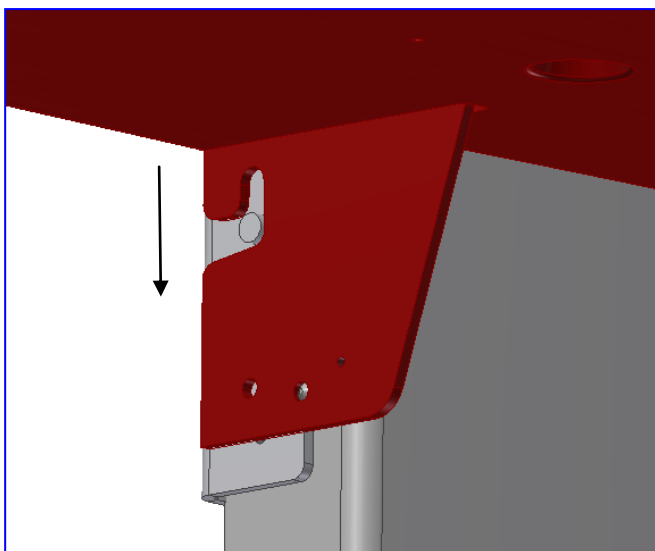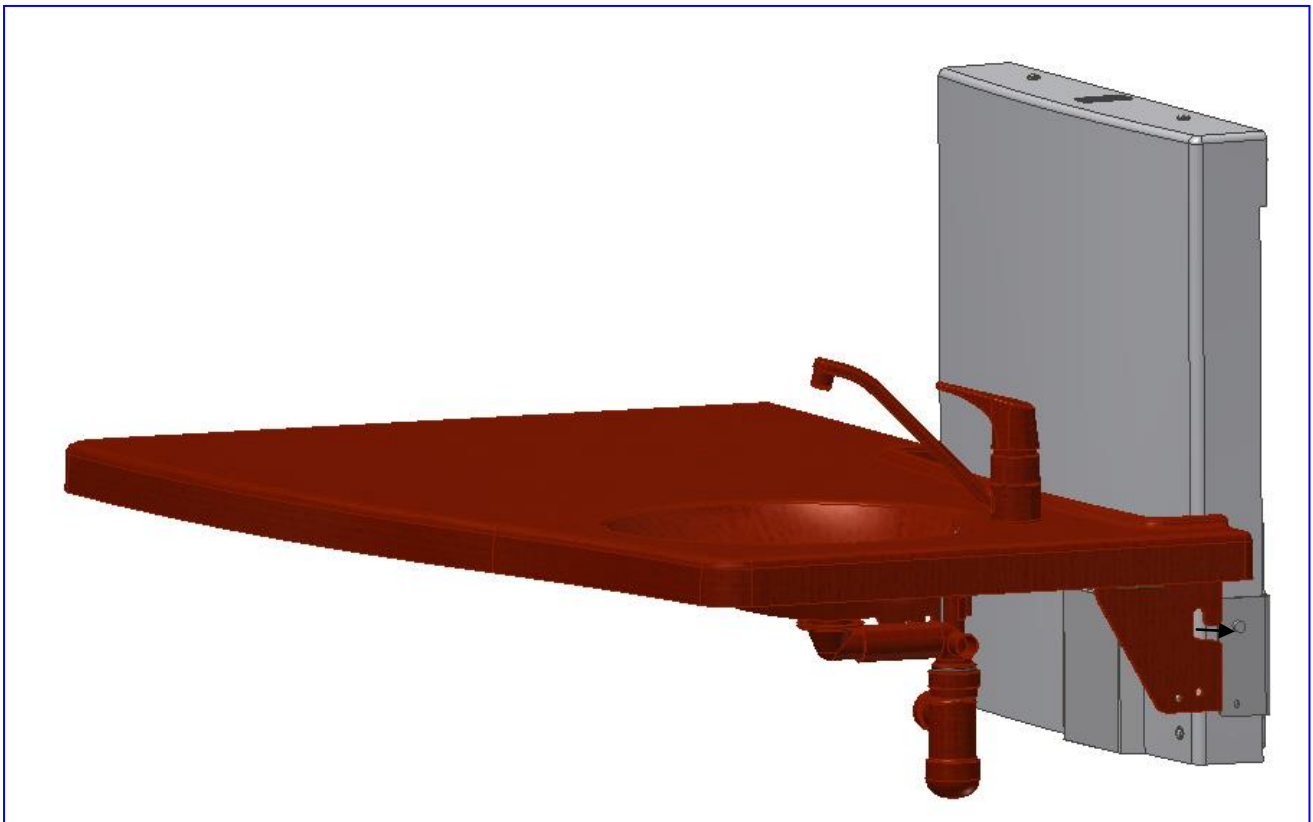3. Mounting of tabletop


Mounting instructions

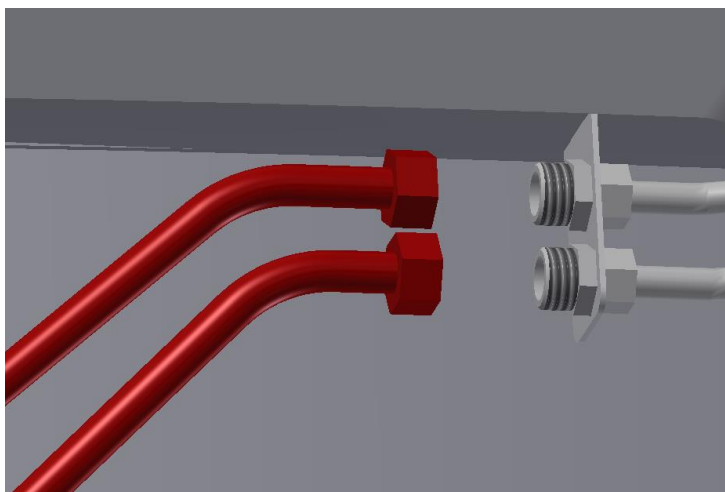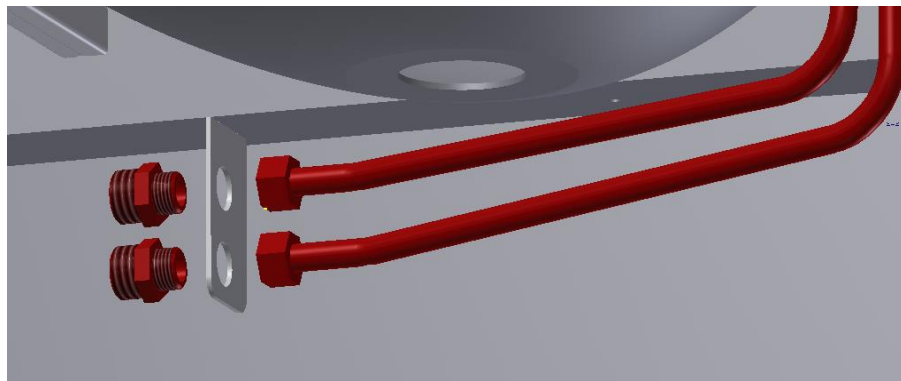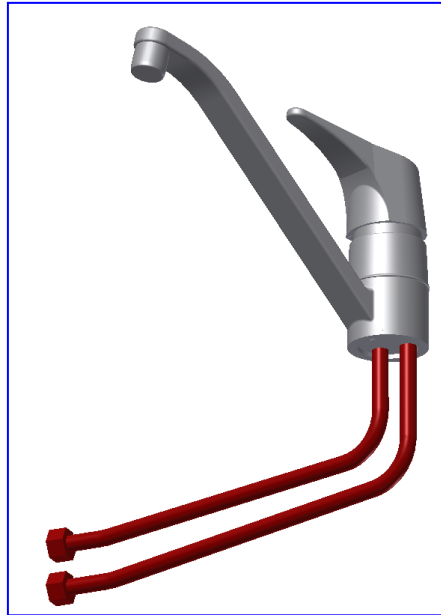4. Connection of water and drain

Mounting instructions

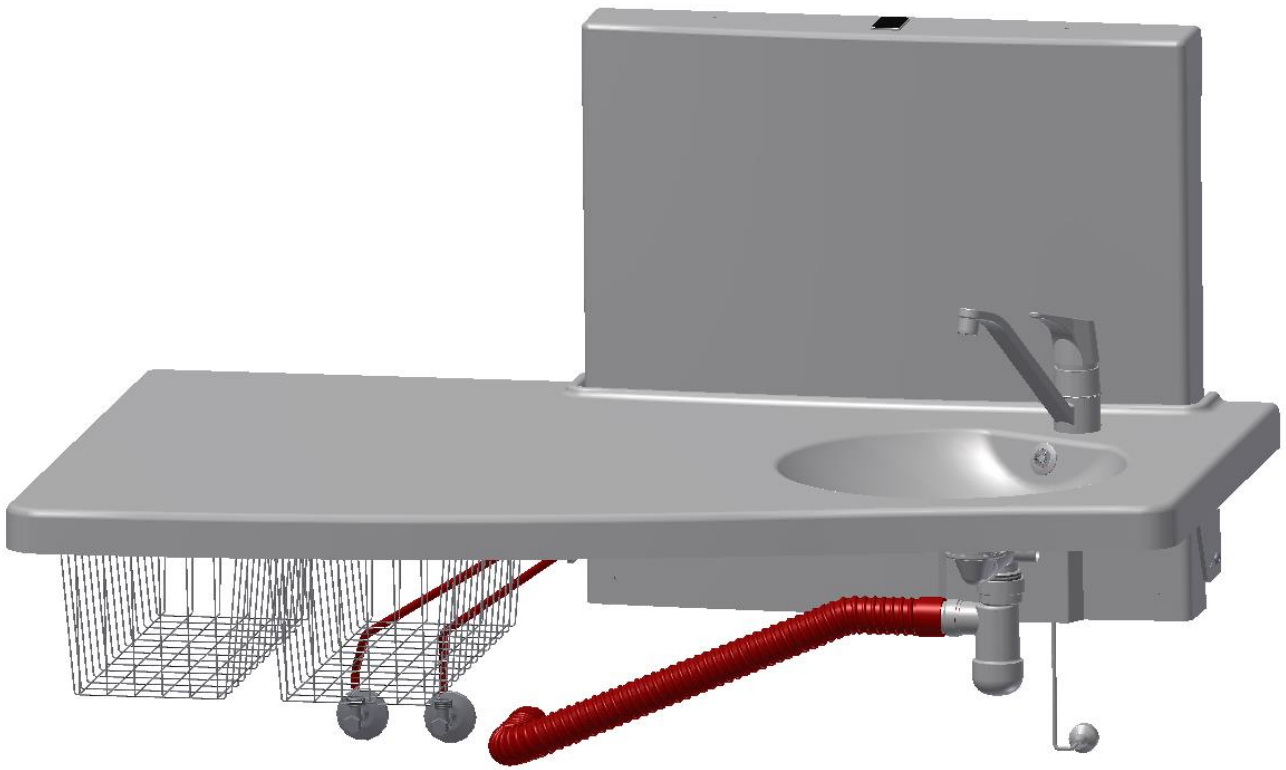If the nursing table is provided with a safety stop, connect the cables between lifting unit and tabletop.

Electrical components

Wiring diagram

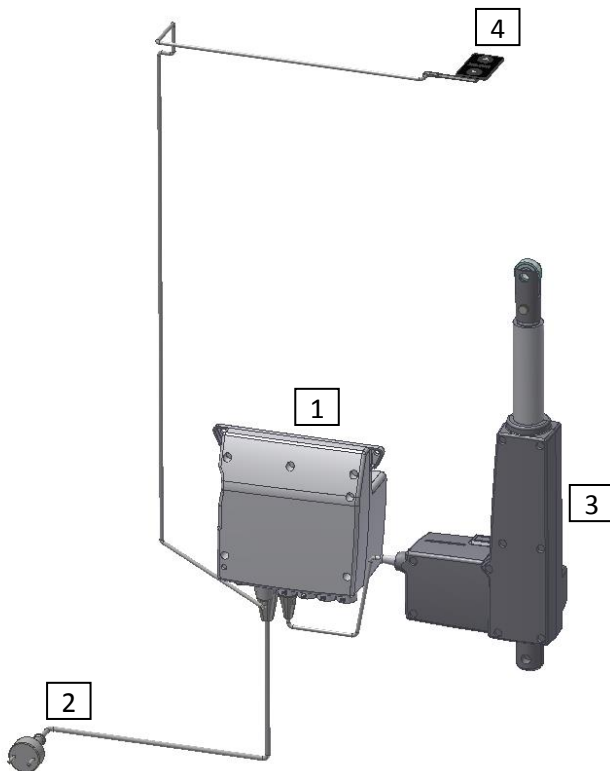
The nursing table is delivered with the control unit mounted on the frame. The diagram shows how to connect the cables with the control unit.

All plug-in connections are waterproof and mounting/dismounting may therefore be a little difficult. Make sure that all plugs have the correct position before inserting them into the control unit. The plugs must be pressed home until the O-ring is no longer visible.

Motor, control unit and hand-control switch have protection IPX6.

The electrical system is designed for periodical use. This means a max. duty cycle of 10%; 2min ON /18min OFF.

- 6. Control unit
- 7. Mains cable 230V
- 8. Motor, to be connected to port 1
- 9. Hand-control switch
- 10. Safety stop cable (optional)

Electrical system for nursing table Medi 2

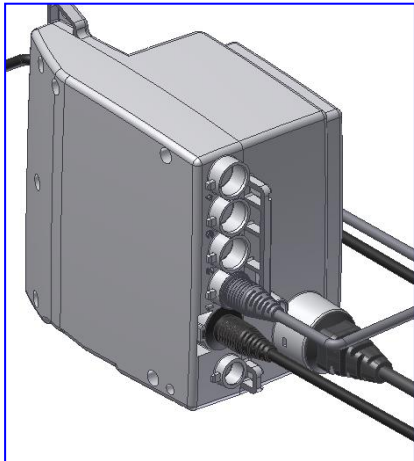
Electrical system for nursing table Medi 2 with optional safety stop under the tabletop

Electrical components

Connections

The nursing table is delivered with the control unit mounted on the frame.

Connect motor cable to port 1. Ports 2, 3 & 4 are not used and have therefore been provided with a blind plug.

Connect hand-control switch to port "HB".

Connect mains cable to the port on the side of the control unit.

Dismounting

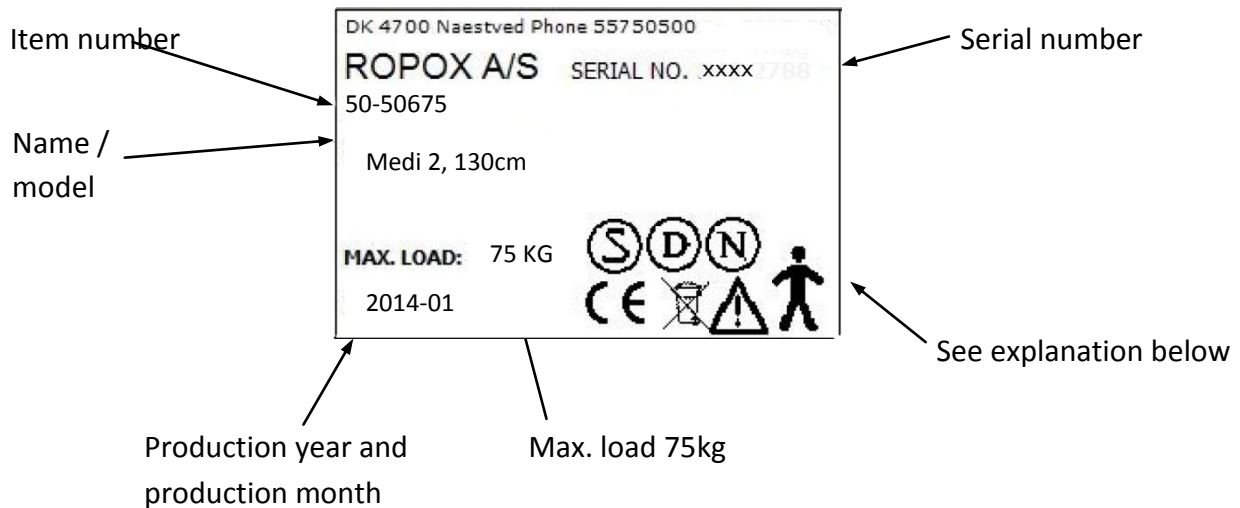
Just pull out the mains cable.

In order to maintain the full IPX6 protection it is very important that all cable connections are correct.

Marking

Label

Each nursing table has a label of its own as illustrated. This label contains important information to be stated in connection with inquiries to the supplier.

Symbols:

The user manual must be read prior to using the product.

Waste must be sorted – if possible for recycling.

Type B equipment.

Class 2 equipment. No connection to earth.

CE marking. This product meets the requirements for General Safety and Functionality of the current EU Directives and Standards.

EMC information

The nursing table meets the requirements for Electromagnetic Compatibility (EMC), if mounting and daily use are in accordance with the instructions of this manual. In seldom cases, the nursing table may be affected by electronic signals from, for instance, laptops, mobile phones or the like.

Support

Fault finding

The nursing table does not move up and down when the hand-control switch is pressed	Check: <ol style="list-style-type: none">6. If 230V has been connected and current is available.7. If all plugs have been correctly inserted.8. If the table only moves upward, check if the safety stop has been blocked/activated.9. Dismount all cables and mount them again.10. Contact Ropox for support.
---	---

Waste handling

Used equipment must be disposed of according to national law. We recommend dismantling of the product to the greatest possible extent so that most parts may be recycled. Note that this product contains electrical components. Examples of sorting groups: Metal, plastic, cables, electronic parts and parts for destruction.

The nursing table may be used at temperatures from 5°C to 45°C

The nursing table may be used at a relative humidity between 5% to 85% non-condensing

Factors affecting the product during storage and transport

Maintenance

- i** Ropox recommends testing the functions of the nursing table and retightening of all bolts and nuts once a year.

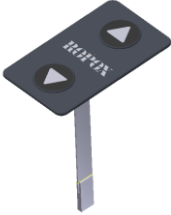
In case of defects, please contact Ropox as soon as possible, and do not use the nursing table until the problem has been solved/remedied.

!! Maintenance of the nursing table should only be carried out by competent personnel !!

If you do not want to carry out the performance test yourself, you may contact Ropox to arrange a visit by a service technician. See contact information at the end of these instructions.

Spare parts

Usual spare parts

97000473	Actuator Type : LA31.3 Stroke 175 mm	
97000434	Control unit Type : CB9096-2	
97000244	Ropox hand-control switch	

Maintenance

Ropox nursing table is maintenance-free. However, for safety reasons we recommend regular inspection of all bolts and moving parts to ensure correct and secure fastening to the wall.

Cleaning

Do not use a high-pressure washer to clean this product.

Simply use a soft cloth or sponge with soapy water of max. 60°C (pH 7) or cleaning alcohol.

Always clean the nursing table after use to prevent dirt from sticking to the surfaces.

Important:

Do not use scouring powder or other abrasives, as they will scratch the surfaces.

Also, avoid the use of cleaning utensils with sharp edges. Never use strong mineral acids or alkaline products.

CE Declaration

EU Declaration of Conformity

I hereby declare under sole responsibility that the following products

50-50675 Nursing Table Medi 2, 130 cm

50-50677 Nursing Table Medi 2, 90 cm

are in conformity with the following directives and standards:

DIRECTIVES

- Council Directive No. 89/392/EEC concerning Machinery amended by Directive No. 98/37/EC
- 2006/95/EEC Low Voltage Directive amended by Directive No. 93/68/EEC
- 2004/108/EEC, EMC Directive amended by Directives Nos. 92/31/EEC and 93/68/EEC
- DS/EN 12221-1:2008, Products for children care – nursing tables for private use – Part 1: Safety requirements

STANDARDS

DS/EN	1041: 2009	DS/EN ISO	60335-1
DS/EN	14971: 2008	DS/EN	12221-1:2008
DS/EN	12182: 2002		

Date: 04-06-2012

Tommy Jensen
Adm. direktør

Complaints

Please see General Terms of Sale and Delivery on www.ropox.com

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Ropox A/S