Loire straight 190, 40-40140-9

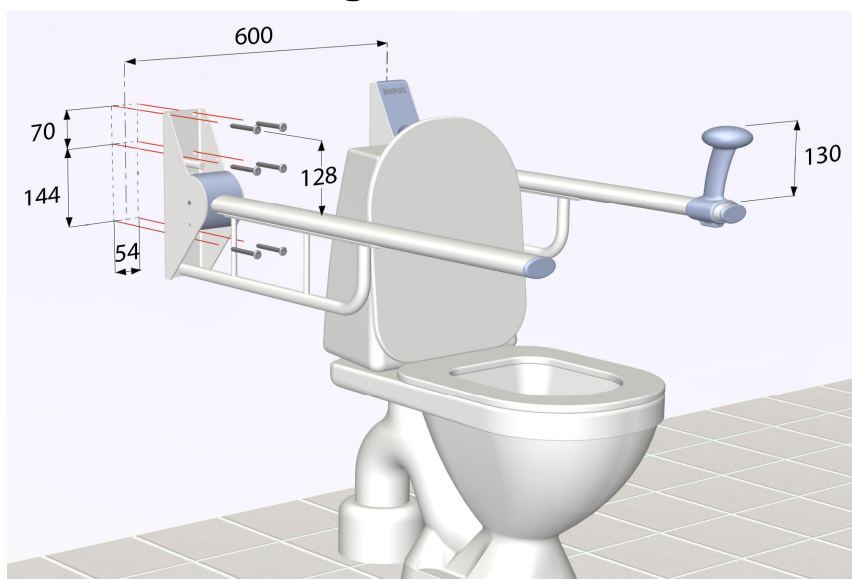
Montageanvisning

Mounting instruction

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I henhold til / according to DS 3028 (Danish Standard)

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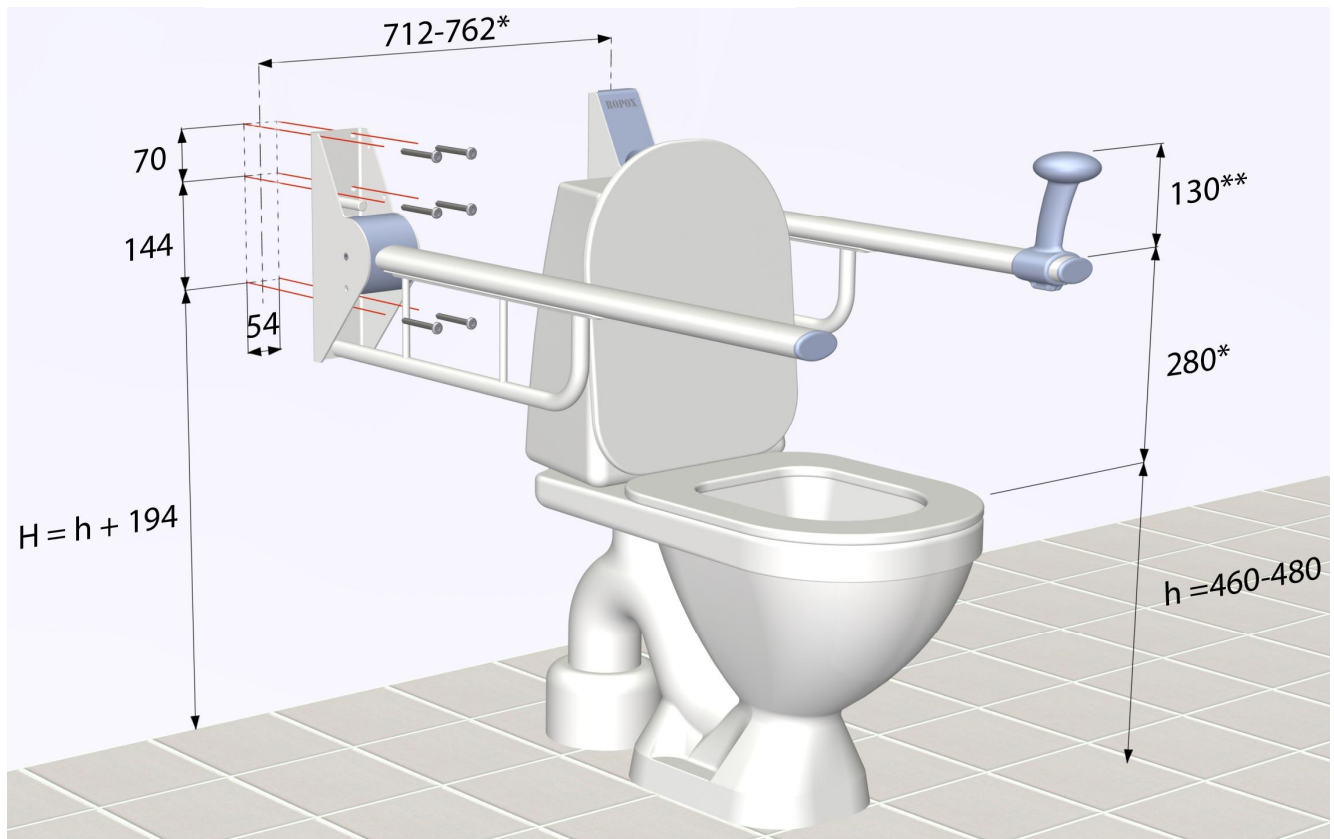
Please find attached mounting template showing position of the 6 holes for easy installation

User heights and installation heights (upper holes in mounting template/bracket)

User height (recommendation)	Installation height (upper holes)	Situation
800 mm (top of support arm)	928 mm	Loire Straight
670 mm (top of support arm) 800 mm (top of Get up/sit down)	798 mm	Loire Straight + Get up/sit down fitting
<p>If any other specific user height is required, simply position the 2 upper holes of the bracket (or the mounting template) 128 mm above the required height of the support arm.</p> <p><u>Example:</u> Required user height is 775 mm Installation height = 775 + 128 mm = 903 mm</p>		

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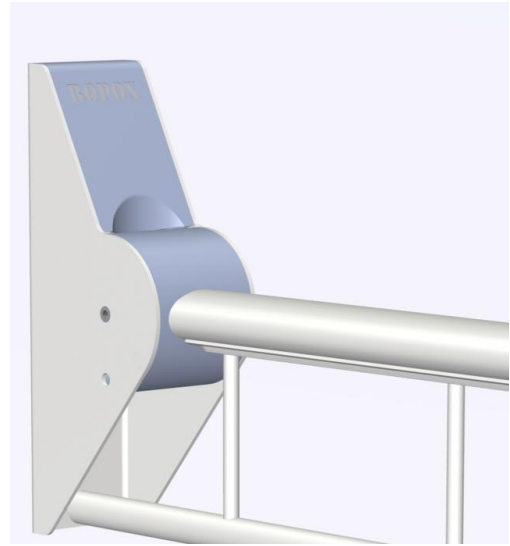
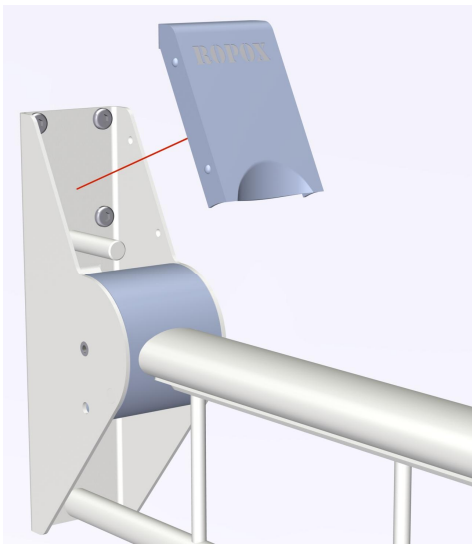
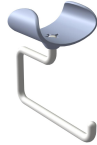
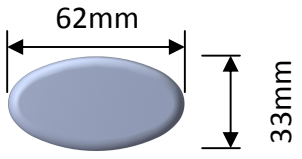
Maintenance

The toilet supports are maintenance free. For safety reasons it is recommendable to check all mounting screws and moving parts to be sure all is tightened ok, and that all attachment to the wall is in correct and safe condition.

Cleaning

To clean this product, do not use a high-pressure washer. This product is not machine washable.

Simply use a soft cloth or sponge with soapy water (pH 7) of max 60°C. Always clean this product after use to prevent dirt from sticking to the surface.

Important:

Do not use scouring powder or other abrasives as they may scratch the surface. Also avoid the use of cleaning utensils with sharp edges. Never use strong mineral acids or alkali.

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Se Generelle salgs- og leveringsbetingelser på www.ropox.dk

Complaints

See General Terms of Sale and Delivery on www.ropox.com

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ROPOX A/S