
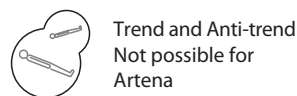
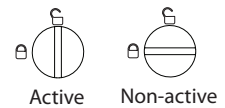


# Artena Care

	Manipulation	Hand switch	Hand switch HD	Lock box ACL
BACKREST				
KNEE REST				
RELAXPOSITION				
HIGH/LOW				
SLEEPING POSITION				
Lock/ Unlock				

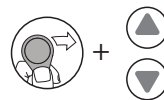

Trend and Anti-trend  
Not possible for  
Artena


The available bed functionality is in line with the chosen and ordered configuration.






## Artena Care


### Quick release

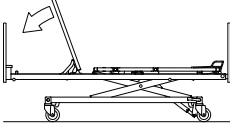

### Lever

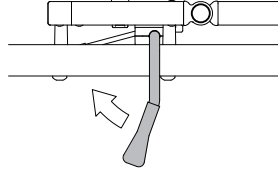

### Bed extension

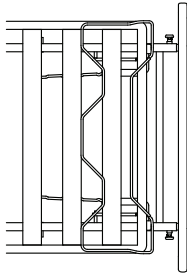







1. lift the footrest
2. pull out both poll knobs
3. pull out the bed extension
4. lock the buttons
5. pull out the foot extension (option)

### Brake mechanism

1 directional castor, fixed

4 castors unbraked

4 castors braked

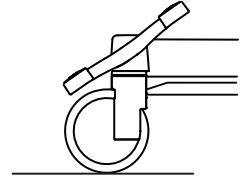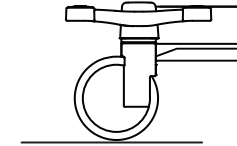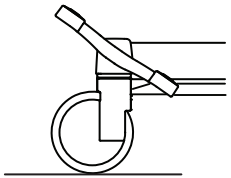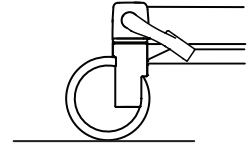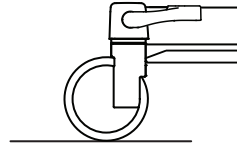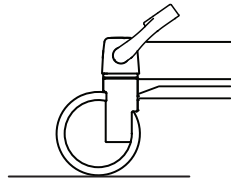

The available bed functionality is in line with the chosen and ordered configuration.

